

GLOVE EASY TURN SLINGS

Instructions For Use

Please read these instructions before use.

INSTRUCTIONS FOR USE

Please read these instructions before use.


PRODUCT CODES & DESCRIPTIONS

All Glove Easy Turn Slings. Product codes prefixed with:

- GTE - Glove Easy Turn

DEVICE DESCRIPTION

The Glove Easy Turn Sling are non-sterile medical devices, complaint with BS EN 10535. Each sling has a sewn woven label containing device Serial Number and Safe Working Load (kg), and a printed label containing all pertinent information.

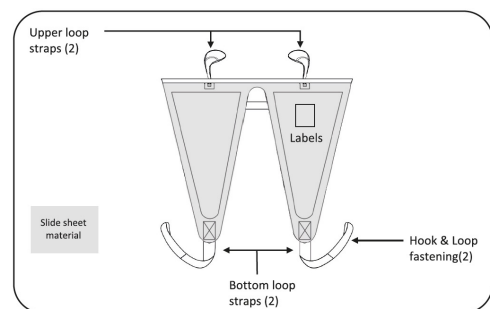

For further information, support and product training requests visit: www.gbukgroup.com

PATIENT INFORMATION

The following information is provided for use by trained healthcare professionals, and carers, and those with the appropriate training or knowledge, however as the learned intermediary between the company and the patient, the trained healthcare professionals and carers, and those with the appropriate training or knowledge must convey the aspects they consider relevant to the individual patient. The patient must be informed of the potential adverse effects & residual risks contained within this document.

PATIENT TARGET GROUP(S)

Intended for patients who require lifting and/or repositioning following a suitable risk assessment.

INTENDED USE

Intended to be used to transfer patients following a risk assessment. The sling should be used in conjunction with a mechanical lifting device (either mobile or ceiling hoist) to ensure safe lifting and repositioning of a patient.

INTENDED USER(S)

The device is intended to be used by trained healthcare professionals and carers, and those with the appropriate training or knowledge. Refer to the Royal College of Nursing Guidelines.

INDICATION

The sling is indicated for use on patients with limited mobility requiring lifting and/or repositioning.

SAFE WORKING LOAD (SWL)

The Glove Easy Turn Sling has a SWL of 200 kg. The SWL is also provided on the label(s) directly attached to the device. The SWL of a sling labelled as patient matched is detailed on the label(s) directly attached to the device. Refer to the device label(s) for applicable SWL of the patient matched sling.

CONTRAINDICATIONS

Do not use on patients whose weight exceeds the safe working load limit of the sling.

WARNINGS & PRECAUTIONS

- A risk assessment must be carried out by a suitably qualified individual before selection and use, to ensure safety for the patient and handler. See 'RISK ASSESSMENT' section.
- Always carry out a visual check of the sling prior to every use for any damage, rips, tears, wearing, loose stitching, material degradation, or any other general signs of wear and tear. The sling should be disposed of in accordance with the disposal instructions, if any of these are present, and/or in the instance of failed LOLER inspection, and a new sling selected.
- Do not use with incompatible hoists.
- Follow local manual handling policy from your organization, National Back Exchange or HSE guidelines.
- All users must be trained or have the appropriate training or knowledge before using the Sling.
- Inspect the device, packaging and labelling prior to use and do not use if damaged.
- Do not use the sling to lift a patient. The Sling is subject to Lifting Operations and Lifting Equipment Regulations (LOLER) Examinations. A LOLER examination must be performed every 6 months, by trained and qualified persons.
- Reusable device. Wash the sling in accordance with the instructions provided in 'CLEANING' section.
- Do not bleach, iron or dry clean the Sling. Do not use stain removers.
- Do not exceed maximum washing and drying temperatures.
- Do not wash below minimum washing temperature.
- Do not place the Glove Easy Turn Sling on hot pipes or heaters. Keep away from flames.
- Where the sling has hooks and loops, ensure any hooks and loops are closed prior to laundering.
- Repeated washing may lead to sling deterioration and/or discolouration to some materials or fading of labels.
- When fitting the sling to a patient in a supine position, ensure to roll the patient towards yourself, a

wall or a bed rail, to reduce the risk and occurrence of falls.

EXPECTED LIFETIME

The expected performance and safety lifetime of the device is 5 years; however slings may last longer. All slings must be LOLER inspected, and the inspection must be within date prior to use. Carry out a risk assessment prior to each use of the sling - Refer to 'WARNINGS & PRECAUTIONS' & 'RISK ASSESSMENT'.

PERFORMANCE CHARACTERISTICS

- 200 kg SWL
- ISO 21856 and ISO 10535 compliant

DIRECTIONS FOR USE

Refer to 'Warnings & Precautions' and 'Risk Assessment' Sections prior to use.

1. The user should lie on their back in the middle of the bed with arms folded across the chest.
2. Gently feed the bottom loop straps through the natural gaps between the user and the bed (waist and behind the knees) to the opposite side.
3. Ease bottom loop straps all the way through and then secure to the bed frame on both sides using the hook and loop fastenings.
4. Attach the upper loop straps to the hoist spreader bar as normal. Perform a 'tug test'* and check for snagging.
5. Slowly raise the hoist until the turn is completed.
6. The spreader bar should only be raised as high as necessary to complete the turn.
7. The user can be held in this position whilst any personal care is carried out.
8. If the user is to remain in this position after turning; ensure they are comfortable, lower the spreader bar, detach the straps from the spreader bar and bed before removing the sling.

*A 'Tug Test' is recommended to ensure straps are properly secured to the spreader bar prior to hoisting. Manually pull on the straps in a downward motion.

POTENTIAL ADVERSE EFFECTS & RESIDUAL RISKS

As with any procedure involving manual handling, use of manual handling aids, and hoisting equipment the typical risks associated apply. Failure of the device during use may expose the patient and user to risks such as falling, and potential subsequent associated injury, and/or user exposure to sudden mechanical stress and potential associated pain and/or musculoskeletal injuries.

RISK ASSESSMENT

A risk assessment and visual inspection must be performed by competent person(s) before using the sling. This should consider the following factors:

- The physical and medical status of the patient, including skin integrity,
- The status of the sling, including the presence of any damage, rips, tears, soil, loose stitching, or any other general signs of wear and tear,
- The compatibility of the hoist and sling, including if the correct size and type of the sling is being used
- The capability of the users, using the equipment; and,
- The situation in which it is being used

WARRANTY

The Glove Easy Turn Sling is covered by a lifetime warranty. Any manufacturing defect will be rectified free of charge. Product damage through wear and tear, neglect or accidental damage is not covered.

CLEANING

Routinely clean the sling, in accordance with these instructions and your facility's infection control protocols. Always follow your organisation's infection control guidelines to determine the appropriate cleaning schedule. Provide a copy of these instructions to the person(s) laundering the sling.

1. Where present, close hook & loop fastenings before washing to prevent damage to the fabric and webbing.
2. Wash gently at a minimum of 40°C and a maximum of 95°C. Use a mild soap solution or a non-biological detergent without bleaching additives.
3. Do not add bleach or stain removers.
4. Air dry, cabinet dry or tumble dry on a cool cycle (below 60°C). Do not tumble dry on a hot setting.

STORAGE CONDITIONS

Standard storage conditions. Keep off the floor. Keep out of direct sunlight. Keep dry.

DISPOSAL

No specific disposal requirements, handle contaminated items as clinical waste. Dispose of device in line with local and/or hospital policy.

COMPLAINTS

Any user who has a complaint or experiences dissatisfaction in the product quality, identity, durability, reliability, safety, usability, effectiveness, and/or performance, should notify the manufacturer and distributor immediately.

In the event of a serious incident involving the device, the user should report the event to the manufacturer (GBUK Group) and the competent authority of the member state in which the user and/or patient is established, immediately, by telephone, email (complaints@gbukgroup.com) or written correspondence.

When filing a complaint, provide the component(s) REF and SN number(s), your name and contact details, and the nature of the complaint.

In the event of a suspected device fault the user should report the event to GBUK Group with the above information and return the device to GBUK Group for examination.

GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire,
YO8 5DD, UK.
T: EU: +44 (0) 1757 288587
E: info@gbukgroup.com
W: www.gbukgroup.com

GBUK GmbH
Wirtschaftsprüfungsgesellschaft
Im Klapperhof 7-23
50670 Köln, Germany

EU REP **QualRep Services BV,**
Utrechtseweg 310 B42,
6812 AR, Arnhem,
The Netherlands
CH REP **Swiss AR Services AG**
Industriestrasse 47,
6300 Zug, Switzerland

