Neo-Cuff<sup>®</sup>

# Blood Pressure Cuff

SINGLE PATIENT USE

**INSTRUCTIONS  
FOR USE**

**GBUK Group Ltd.**  
Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK  
**EU: +44 (0) 1757 282 945**  
**US: 18665 289 422**  
[www.gbukhealthcare.com](http://www.gbukhealthcare.com)  
[info@gbukhealthcare.com](mailto:info@gbukhealthcare.com)

**QualRep Services BV**  
Utrechtseweg 310 B42  
6812 AR Arnhem,  
The Netherlands

	<b>Page</b>
<b>EN</b> Neo-Cuff Blood Pressure Cuff Single Patient Use	4
<b>FR</b> Brassard De Tensiomètre Neo-Cuff Destiné À Être Utilisé Sur Un Seul Patient	5
<b>IT</b> Bracciale Per La Misurazione Della Pressione Sanguigna Neo-Cuff Per Uso Individuale	6
<b>ES</b> Manguito Neo-Cuff Para Uso En Un Único Paciente	7
<b>DE</b> Neo-Cuff-Blutdruckmanschette Zur Verwendung An Einem Einzelnen Patienten	8
<b>DA</b> Neo-Manchet Blodtryksmanchet Bruges Til En Patient.	9
<b>NO</b> Blodtrykksmansjett Fra Neo-Cuff Til Engangsbruk	10
<b>SE</b> Neo-Cuff Blodtrykksmanschett Enpatientsbruk	11
<b>NL</b> Bloeddrukmetermanschet Neo-Cuff™ Voor Gebruik Bij Één Patiënt	12
<b>PL</b> Mankiet Do Pomiaru Ciśnienia Krwi Neo-Cuff – Do Użycia U Jednego Pacjenta	13
<b>PT</b> Manguito De Tensão Arterial Neo-Cuff De Utilização Em Paciente Único	14
<b>EL</b> Περιχειρίδα Πιεσόμετρου Neo-Cuff Για Χρήση Σε Έναν Ασθενή	15
<b>TR</b> Neo-Cuff Kan Basıncı Manşeti, Tek Hastada Kullanım İçin	16
<b>CS</b> Manžeta K Měření Tlaku Krve Neo-Cuff K Použití Pro Jednoho Pacienta	17
<b>ET</b> Vererõhumansett Neo-Cuff Kasutamiseks Ühel Patsiendil	18
<b>HU</b> Neo-Cuff Vérnyomásmérõ Madzsetta, Egyedüli Használatra.	19
<b>LV</b> Neo-Cuff Asinsspiediena Mērišanas Aproce Lietošanai Vienam Pacientam	20
<b>MT</b> Apparat Ghat-Tehid Tal-Pressjoni Neo-Cuff Li Jintuża Għal Pazjent Wieħed Biss	21
<b>RO</b> Manșetă De Unică Folosință Pentru Măsurarea Tensiunii Arteriale Neo-Cuff	22
<b>SL</b> Neo-Cuff Manšete Za Merjenje Krvnega Tlaka Za Uporabo Pri Enem Samem Pacientu	23

## Neo-Cuff Blood Pressure Cuff Single Patient Use

### Intended use

Blood pressure cuff, intended for use in conjunction with non-invasive blood pressure measurement systems.

### INDICATIONS FOR USE

For connection to a sphygmomanometer or patient monitoring device capable of neonatal mode blood pressure measurement.

### DESCRIPTION

Neo-Cuff® Blood Pressure Cuffs are an inflatable device made from soft-feel material, and intended for skin surface contact. Markings show the cuff size, limb circumference, artery position and cuff range. Available in a range of sizes.

### PATIENT TARGET GROUP

Neonatal patients requiring blood pressure monitoring

### INTENDED USERS

The device is intended to be used by a trained healthcare professional and others with appropriate training and knowledge.

### CONTRA-INDICATIONS

Do not connect to any other patient circuit, or medical gas or fluid supply or drain. Do not apply cuff to wounded areas on patient, or areas where skin is delicate or damaged.

### WARNINGS AND PRECAUTIONS

- This device is intended for single patient use.
- Do not over tighten cuff.
- Minimize cuff movement and limb motion during readings.
- Ensure that the hose is not kinked or compressed.
- Inspect blood pressure cuffs for damage or contamination before each use.
- Do not use the blood pressure cuff if it is damaged or contaminated.
- Observe the range markers on each blood pressure cuff.
- Always use the appropriate size cuff for each patient.
- In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

### OPERATING INSTRUCTIONS

1. Measure the circumference of the patient's limb and select the appropriate size cuff.
2. Apply the cuff to the arm (or thigh) so that the "Artery" arrow on the cuff is over the brachial artery (or femoral artery).

3. Wrap the cuff around the arm (or thigh). When folding the cuff around the arm (or thigh) ensure that the index line falls within the range markers. If it does not, then use a smaller or larger cuff as indicated by the guidance markings on the cuff.
4. Ensure the cuff is snugly fitted and secured by means of the hook-and-loop fastenings. It should be possible to insert two fingers between the patient and the cuff.
5. Connect the hose(s) on the cuff to the measuring equipment ensuring proper engagement of the connectors.
6. Refer to the operating instruction of the measurement device to measure the patient's blood pressure.
7. After use, when the cuff is deflated, disconnect the cuff from the measuring device. Unwrap the cuff from the patient's arm (or thigh). Express residual air from the cuff by pressing it flat. The cuff may be folded for storage.
8. If after use cleaning is required, follow cleaning instructions below.

### STORAGE AND HANDLING

Do not use if packaging is damaged. Keep out of direct sunlight. Keep dry

### DISPOSAL PROCEDURES

The device must be discarded after use in accordance with local waste guidelines.

### CLEANING

The blood pressure cuff may be wiped clean using appropriate surface cleaning agents and methods used in the healthcare setting. Ensure that local policies are observed. Do not allow fluid to enter the hoses.

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

\*Trademark or Registered Trademark of  
GBUK Group Ltd.  
[www.gbukgroup.com](http://www.gbukgroup.com)

Symbol glossary available at [www.gbukhealthcare.com](http://www.gbukhealthcare.com).

**GBUK Group Ltd.**

Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK

**EU: +44 (0) 1757 288 587**

**US: 18665 289 422**

[info@gbukhealthcare.com](mailto:info@gbukhealthcare.com)

**QualRep Services BV**

Utrechtseweg 310 B42  
6812 AR Arnhem,  
The Netherlands