Bio-Cuff™

Blood Pressure Cuff

SINGLE PATIENT USE

**INSTRUCTIONS
FOR USE**

GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0) 1757 282 945
US: 18665 289 422
www.gbukhealthcare.com
info@gbukhealthcare.com

QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

	Page
EN Bio-Cuff Blood Pressure Cuff Single Patient Use	4
FR Brassard De Tensiomètre Bio-Cuff Destiné À Être Utilisé Sur Un Seul Patient	5
IT Bracciale Per La Misurazione Della Pressione Sanguigna Bio-Cuff Per Uso Individuale	6
ES Manguito Bio-Cuff Para Uso En Un Único Paciente	7
DE Bio-Cuff-Blutdruckmanschette Zur Verwendung An Einem Einzelnen Patienten	8
DA Bio-Manchet Blodtrykksmanchet Er En Enhed Til Engangsbrug På En Patient.	9
NO Blodtrykksmansjett Fra Bio-Cuff Til Engangsbruk	10
SE Bio-Cuff Blodtrycksmanschett Enpatientsbruk	11
NL Bloeddrukmetermanchet Bio-Cuff™ Voor Gebruik Bij Één Patiënt	12
PL Mankiet Do Pomiaru Ciśnienia Krwi Bio-Cuff – Do Użycia U Jednego Pacjenta	13
PT Manguito De Tensão Arterial Bio-Cuff De Utilização Em Paciente Único	14
EL Περιχειρίδα Πιεσόμετρου Bio-Cuff Για Χρήση Σε Έναν Ασθενή	15
TR Bio-Cuff Kan Basıncı Manşeti, Tek Hastada Kullanım İçin	16
CS Manžeta K Měření Tlaku Krve Bio-Cuff K Použití Pro Jednoho Pacienta	17
ET Vererõhumansett Bio-Cuff Kasutamiseks Ühel Patsiendil	18
HU Bio-Cuff Vérnyomásmérõ Madzsetta, Egyedüli Használatra.	19
LV Bio-Cuff Asinsspiediena Mõrõšanas Aproce Lietõšanai Vienam Pacientam	20
MT Apparat Ghat-Tehid Tal-Pressjoni Bio-Cuff Li Jintuza Ghal Pazjent Wiehed Biss	21
RO Manşetă De Unică Folosință Pentru Măsurarea Tensiunii Arteriale Bio-Cuff	22
SL Bio-Cuff Manšete Za Merjenje Krvnega Tlaka Za Uporabo Pri Enem Samem Pacientu	23

Bio-Cuff Blood Pressure Cuff Single Patient Use

Intended use

Blood pressure cuff intended for use in conjunction with non-invasive blood pressure measurement systems.

INDICATIONS FOR USE

For connection to a sphygmomanometer or patient monitoring device to enable the measurement of blood pressure.

DESCRIPTION

Bio-Cuff® Blood Pressure Cuffs are an inflatable device made from soft-feel material and intended for skin surface contact. Markings show the cuff size, limb circumference, artery position and cuff range. Available in a range of sizes and industry-standard connectors.

PATIENT TARGET GROUP

Paediatric and adult patients requiring blood pressure monitoring.

INTENDED USERS

The device is intended to be used by a trained healthcare professional and others with appropriate training and knowledge.

CONTRA-INDICATIONS

Do not connect to any other patient circuit, or medical gas or fluid supply or drain.

Do not apply cuff to wounded areas on patient, or areas where skin is delicate or damaged

WARNINGS AND PRECAUTIONS

- This device is intended for single patient use.
- Do not over tighten cuff.
- Minimize cuff movement and limb motion during readings.
- Ensure that the hose is not kinked or compressed.
- Inspect blood pressure cuffs for damage or contamination before each use.
- Do not use the blood pressure cuff if it is damaged or contaminated.
- Observe the range markers on each blood pressure cuff.
- Always use the appropriate size cuff for each patient
- In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

OPERATING INSTRUCTIONS

1. Measure the circumference of the patient's limb and select the appropriate cuff size.
2. Apply the cuff to the arm (or thigh) so that the "Artery" arrow on the cuff is over the brachial artery (or femoral artery).

3. Wrap the cuff around the arm (or thigh). When folding the cuff around the arm (or thigh) ensure that the index line falls within the range markers. If it does not then use a smaller or larger cuff as indicated by the guidance markings on the cuff.
4. Ensure the cuff is snugly fitted and secured by means of the hook-and-loop fastenings. It should be possible to insert two fingers between the patient and the cuff.
5. Connect the hose(s) on the cuff to the measuring equipment ensuring proper engagement of the connectors.
6. Refer to the operating instruction of the measurement device to measure the patient's blood pressure.
7. After use, when the cuff is deflated, disconnect the cuff from the measuring device. Unwrap the cuff from the patient's arm (or thigh). Express residual air from the cuff by pressing it flat. The cuff may be folded for storage.
8. If after use cleaning is required, follow cleaning instructions below.

STORAGE AND HANDLING

Do not use if packaging is damaged. Keep out of direct sunlight. Keep dry

DISPOSAL PROCEDURES

The device must be discarded after use in accordance with local waste guidelines.

CLEANING

The blood pressure cuff may be wiped clean using appropriate surface cleaning agents and methods used in the healthcare setting. Ensure that local policies are observed.

Do not allow fluid to enter the hoses.

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

*Trademark or Registered Trademark of
GBUK Group Ltd.
www.gbukgroup.com

Symbol glossary available at www.gbukhealthcare.com.

GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0) 1757 288 587
US: 18665 289 422
info@gbukhealthcare.com

QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

GB-SP-03 2306 17