



## Grip-Lok 3300M for Securement of Surgical Jejunostomy Device/PEG Tubes


Prepare the skin according to the standard hospital protocol for dressing application.

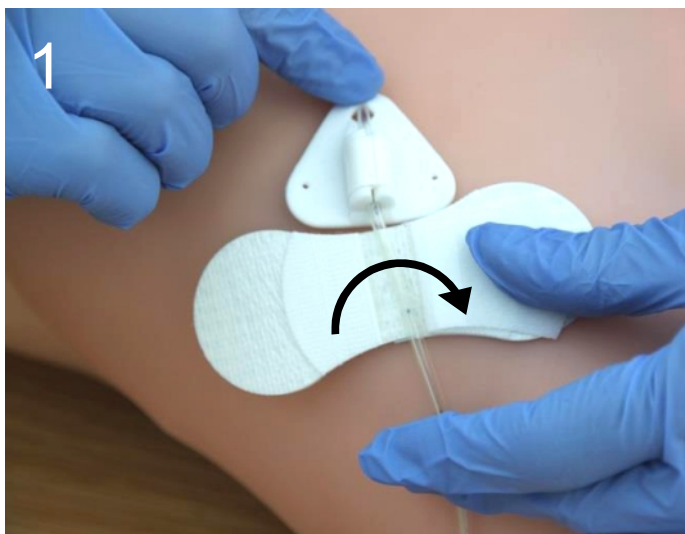


Open the top flap and remove the inside liner. Slide the Grip-Lok® under the jejunostomy/PEG tube.

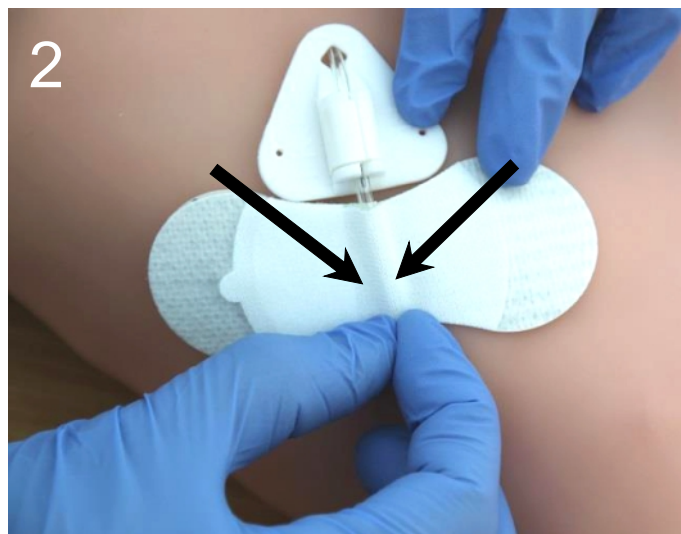

Firmly secure the jejunostomy/PEG tube by closing the top flap and then pinching around and along the length of the tube.

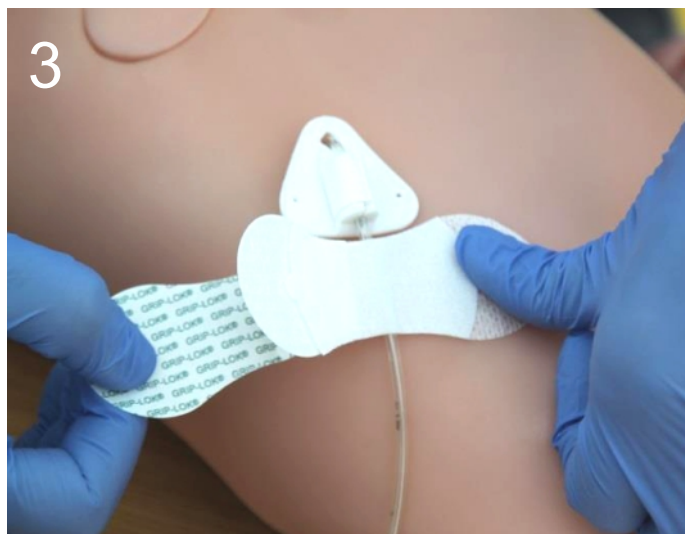

Hold the Grip-Lok® and tube in position by placing thumb on the tabbed side, away from the centre section. Pull the lower release liner on the non-tabbed side and adhere to the skin.

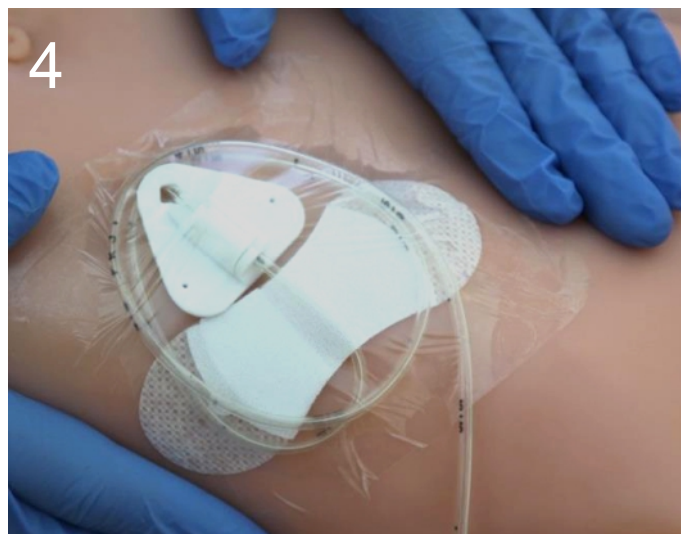

Holding the Grip-Lok® on the tab side, remove the remaining release liner and secure to the skin. Then gently press down around the edges and top surface of the Grip-Lok® to ensure the adhesive connects well to the skin and to the tube. **OPTIONAL: FILM DRESSING (TEGADERM) CAN BE APPLIED.**