



# APPLICATION GUIDE

## Grip-Lok 3300M for Securement of PEG Tubes


Prepare the skin according to the standard hospital protocol for dressing application.

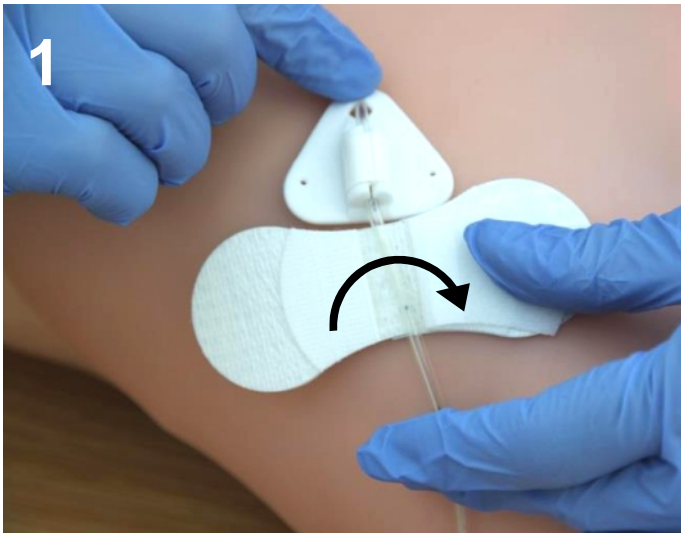

Open the top flap and remove the inside liner. Slide the Grip-Lok® under the PEG tube.

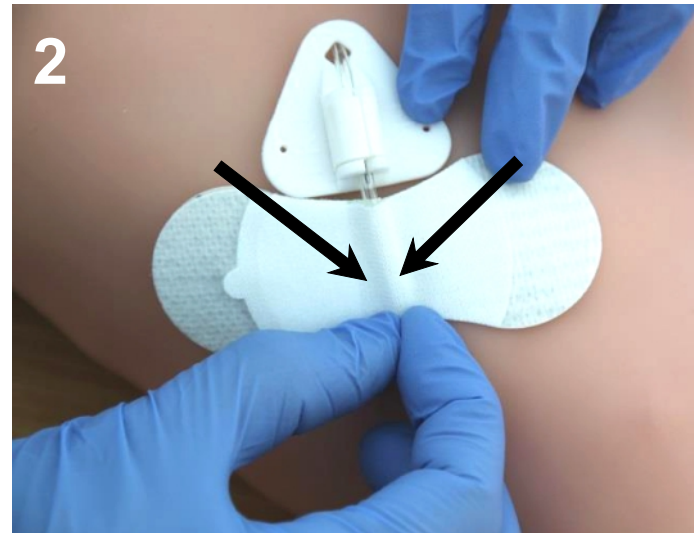

Firmly secure the PEG tube by closing the top flap and then **pinching around and along the length of the tube.**

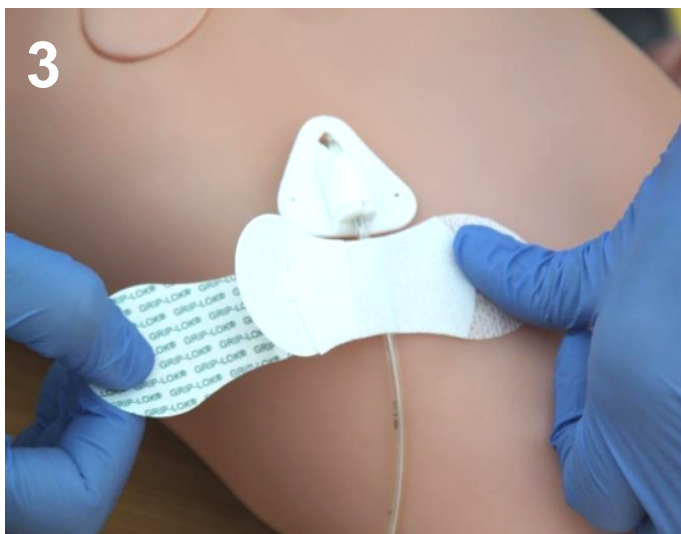

Hold the Grip-Lok® and tube in position by placing thumb on the tabbed side, away from the centre section. Pull the lower release liner on the non-tabbed side and adhere to the skin.



Holding the Grip-Lok® on the tab side, remove the remaining release liner and secure to the skin. Then gently press down around the edges and top surface of the Grip-Lok® to ensure the adhesive connects well to the skin and to the tube.