


APPLICATION GUIDE


Grip-Lok 3300M for Securement of Surgical Jejunostomy Device

(Available on prescription using PIP Code: 368-7258)


Prepare the skin according to the standard hospital protocol for dressing application.

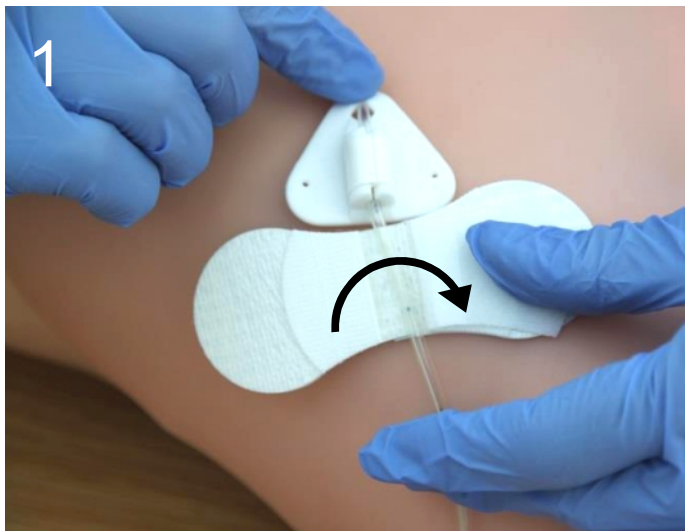


Open the top flap and remove the inside liner. Slide the Grip-Lok® under the jejunostomy tube.

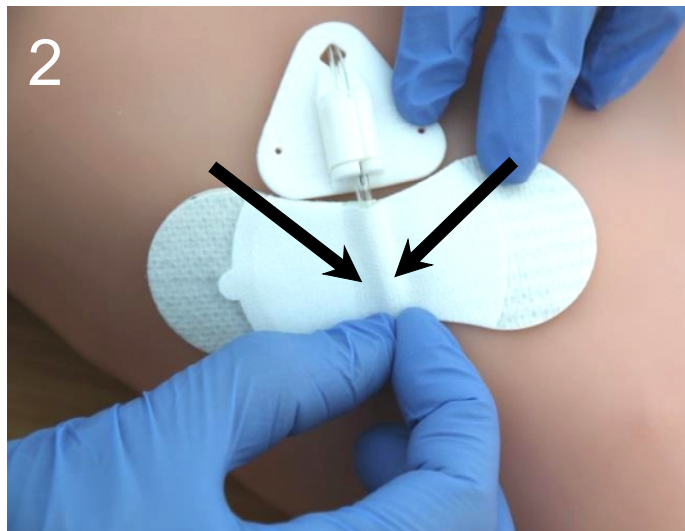

Firmly secure the jejunostomy tube by closing the top flap and then **pinching around and along the length of the tube.**

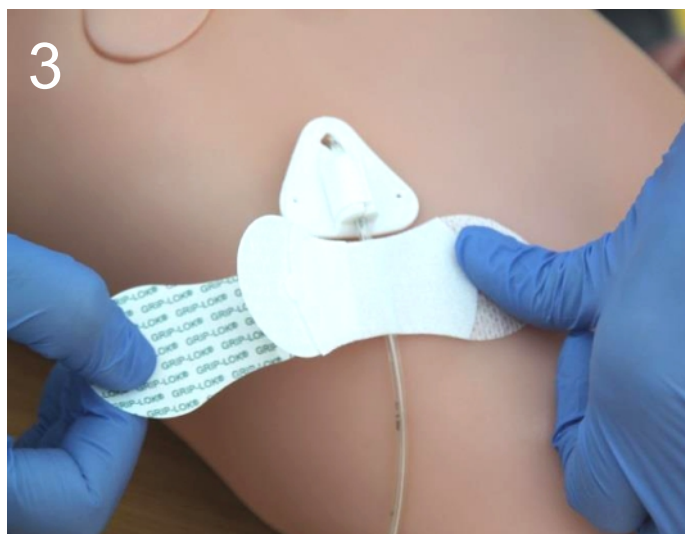

Hold the Grip-Lok® and tube in position by placing thumb on the tabbed side, away from the centre section. Pull the lower release liner on the non-tabbed side and adhere to the skin.

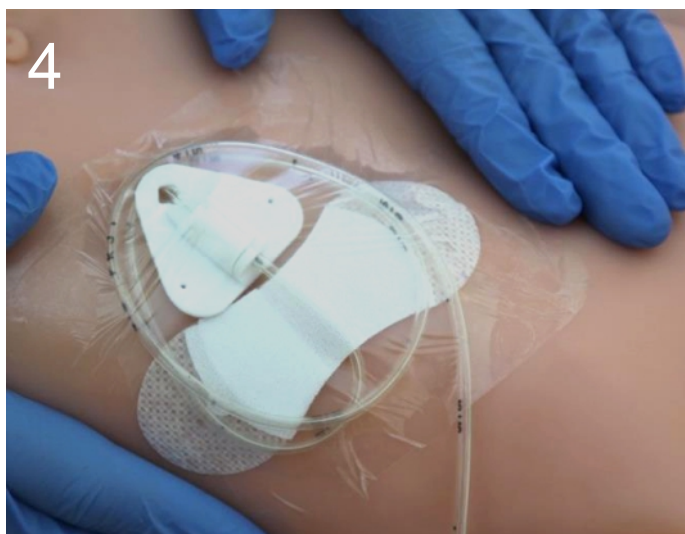

Holding the Grip-Lok® on the tab side, remove the remaining release liner and secure to the skin. Then gently press down around the edges and top surface of the Grip-Lok® to ensure the adhesive connects well to the skin and to the tube. **OPTIONAL: FILM DRESSING (TEGADERM) CAN BE APPLIED.**



Grip-Lok® NHS Supply Chaincode: ELW704
Available on prescription (FP10)code:3300M-10PK

U_e! !! +* QW! % '&&! ^ r}, 'f..t val yepw| }f,} { %pqtvoq! &# V# %% g%