

Orbit Positioning Guide



1. Intoprone blue handles face up. Wider handles at head end of trolley/bed.



2. Out of Prone red handles face down.



3. Patient shoulder lined up with right angles of stitching at head end of Intoprone.



4. Line up the top of Out of Prone with the shoulder stitching line of Intoprone.



5. Using the handles on the edges, slide patient over so the pivoting shoulder is in the correct position before turning.



6. Staff on the 'pulling side' need their handle on top.



Scan for more information & order codes
www.gbukbanana.com



Scan the QR to contact your local Area Business Manager:

Email: info@gbukbanana.com
Main: +44 (0)1757 288 587