




Scan me!

Urimeter 500

Quick Set Up Guide


Step 1

Connect the male tip of the collection chamber to the female port of the drainage bag

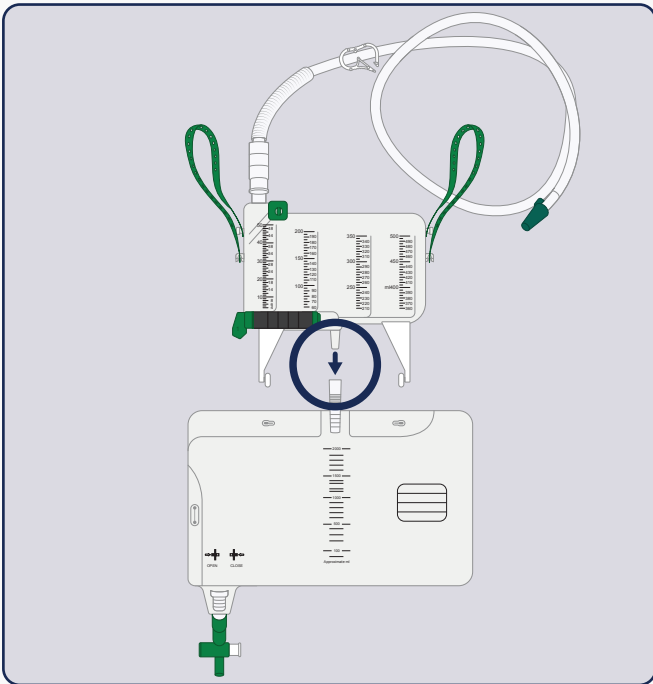


Step 2

Thread the holes of the drainage bag through the two hooks on the bottom of the collection chamber

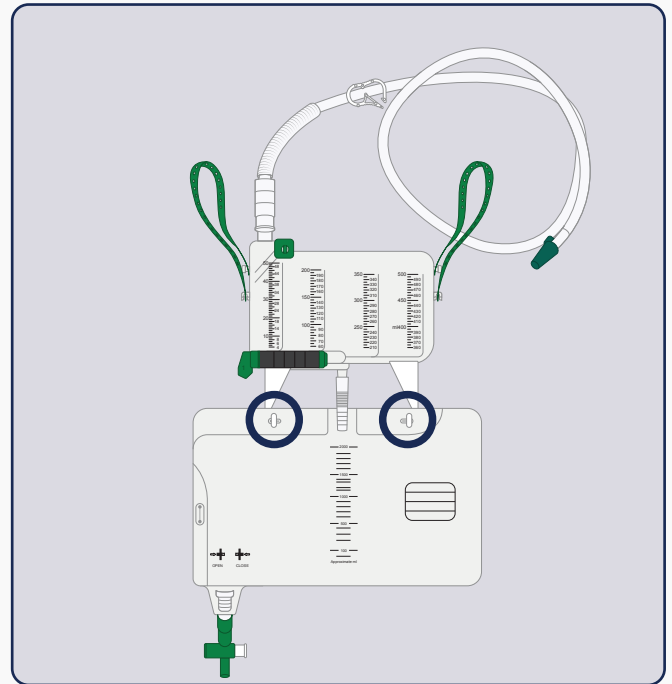


Step 3

Check the drainage bag is not bent/folded

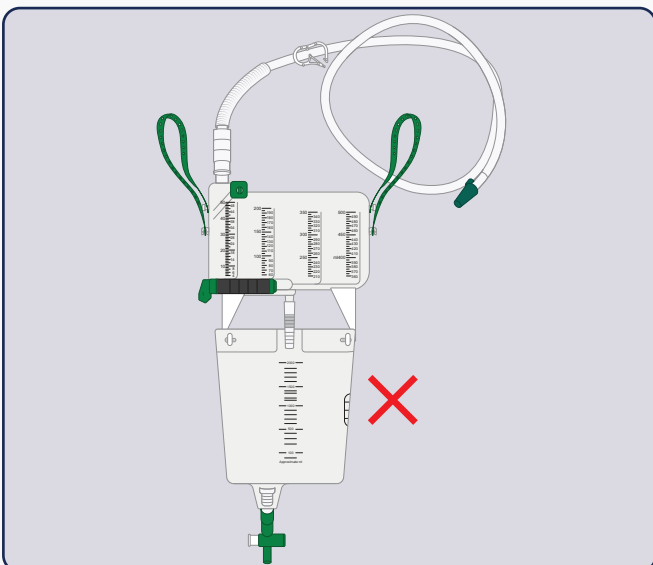

Step 4

Hang Urimeter to side rail using straps

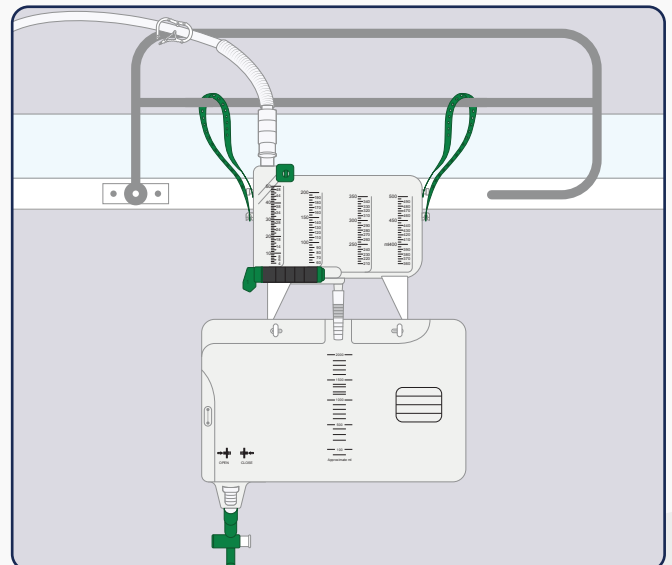




Scan me!

Urimeter 500

Quick Set Up Guide


Step 5

Close the outlet tap on the collection chamber and on the urine bag

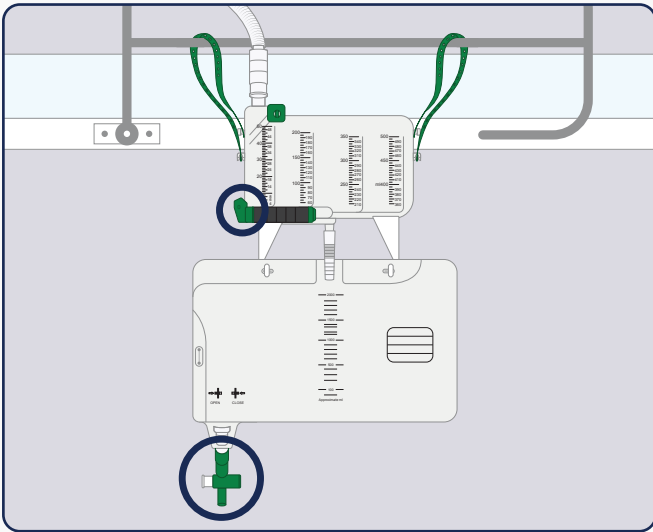

Step 6

Feed outlet tap into hole on urine bag

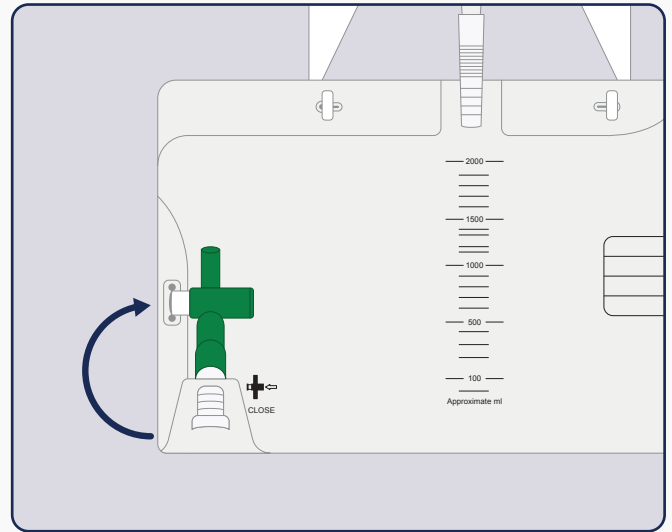

Step 7

Insert connection tubing to catheter
– check the connection is secure

