Drainage Bags

EVA & PVC

INSTRUCTIONS FOR USE

	Page
EN Enteral Drainage Bags EVA & PVC	4
FR Poches de drainage Entéral en EVA et PVC	6
IT Sacche per drenaggio Enterale EVA & PVC	8
ES Bolsas de drenaje Entérico de EVA y PVC	10
DE Enterale Drainagebeutel, EVA und PVC	12
DA Enteral drænposer EVA & PVC	14
NO Enterale drenasjeposer EVA og PVC	16
SV Enterala dränagepåsar EVA och PVC	18
NL Enterale drainagezak EVA & PVC	20
PL Worki do drenażu dojelitowego EVA i PCV	22
PT Sacos de drenagem entérica EVA e PVC	24
EL Εντερικοί σάκοι παροχέτευσης EVA & PVC	26
TR Enteral Drenaj Torbası EVA ve PVC	28
CS Enterální drenážní vaky z EVA a PVC	30
ET EVA & PVC-materjalist enteraalised drenaažikotid	32
HU (Enteral)Emésztőrendszeri Folyadékgyűjtő Zsák EVA & PVC	34
LV Enterālās drenāžas maisi EVA un PVC	36
MT Borża għat-Tbattil tal-Intestini EVA & PVC	38
RO Saci de drenaj enteral EVA și PVC	40
SL Enteralna drenažna vrečka EVA in PVC	42

GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0) 1757 282 945
US: 18665 289 422
www.gbukenteral.com
info@gbukenteral.com

QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

EN

Enteral Drainage Bags EVA & PVC

Instructions for Use

Please read these instructions before use.

DEVICE DESCRIPTION

An Enteral ISOSAF ENFit drainage bag is a single use medical device. The EVA Drainage bags are sterilised using Ethylene Oxide. The PVC Drainage bags are provided non sterile. Enteral Drainage Bags are fitted with ENFit safety connectors in accordance with ISO 80369-3. Enteral Drainage Bags are DEHP and latex free. Measuring graduations are printed on each Enteral Drainage Bag. Intended to be used by patients requiring gastric drainage.

INTENDED USE

Enteral Drainage Bags are designed for the passive drainage of gastric fluid.

INTENDED USER

The device is intended to be used by a trained healthcare professional.

PATIENT POPULATION

Intended for patients requiring passive drainage of gastric fluid.

INDICATIONS FOR USE

Enteral Drainage Bags are designed to be used only with feeding/drainage tubes that are fitted with ENFit connectors in accordance with ISO 80369-3.

CONTRAINDICATIONS

The device should not be used for high volume decompression in emergency situations and/or decompression of highly viscous fluid other than gastric secretions.

WARNINGS AND PRECAUTIONS

The device is single use only. Reuse may result in loss of device performance and/or infection.

Once the drainage bag is disconnected from the feeding tube, it should be discarded.

Do not use for re-feeding.

Do not leave the drainage bags full. The bag should be emptied when nearing its capacity to avoid inhibition of drainage.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

DIRECTIONS

1. Before connection of the Enteral Drainage Bag ensure that the cap on the distal end of the drainage bag is closed.
2. Attach the ENFit female connector on the proximal end of the Enteral Drainage Bag to the feeding/drainage tube. Twist the connectors together ½ turn clockwise to lock.
3. Hang or store enteral drainage bag below the patients stomach level to ensure the stomach contents can flow easily into the bag via gravity.
4. Empty the Enteral Drainage Bag every four to eight hours, more frequently if the bag becomes full. **Do not leave the drainage bags full. The bag should be emptied when nearing its capacity to avoid inhibition of drainage.**
5. Empty the Enteral Drainage Bag by opening the distal connector on the drainage bag. It is not necessary to disconnect the drainage bag from the feeding/drainage tube to empty.
6. Wash hands with soap and water before and after touching the Enteral Drainage Bag.
7. Check the drainage bag tubing periodically to make sure the tubing is not kinked, or that the bag is not squashed.
8. To disconnect the Enteral Drainage Bag from the feeding/drainage tube hold the ENFit male connector of the feeding/ drainage tube and twist anticlockwise the ENFit female connector of the Enteral Drainage Bag until it becomes loose.
9. Dispose of the Enteral Drainage Bag in accordance with local policy.

CLEANING & DISINFECTION

N/A - Single use device. Dispose after use.

STORAGE CONDITIONS

Do not use if packaging is damaged. Keep out of direct sunlight. Keep dry.

DISPOSAL

The device and contents must be discarded after use in accordance with local waste guidelines.

AVAILABLE PRODUCTS

EVA ENTERAL DRAINAGE BAGS

- DBISO-50** EVA Drainage Bag with short inlet tube with female ENFit® ISO 80369-3 connector and male ENFit® port & cap/ tether **50ml**
- DBISO-100** EVA Drainage Bag with short inlet tube with female ENFit® ISO 80369-3 connector and male ENFit® port & cap/ tether **100ml**
- DBISO-250** EVA Drainage Bag with short inlet tube with female ENFit® ISO 80369-3 connector and male ENFit® port & cap/ tether **250ml**
- DBISO-500** EVA Drainage Bag with 50cm inlet tube with female ENFit® ISO 80369-3 connector and funnel connector to outlet **500ml**
- DBISO-1000** EVA Drainage Bag with 50cm inlet tube with female ENFit® ISO 80369-3 connector and funnel connector to outlet **1000ml**
- DBISON-500** EVA Drainage Bag with short inlet tube with female ENFit® ISO 80369-3 connector and male ENFit® port and cap/ tether **500ml**

PVC ENTERAL DRAINAGE BAGS

- DBP-100** PVC Drainage Bag with ENFit® ISO 80369-3 connector **100ml**
- DBP-250** PVC Drainage Bag with ENFit® ISO 80369-3 connector **250ml**
- DBP-500** PVC Drainage Bag with ENFit® ISO 80369-3 connector **500ml**
- DBP-1000** PVC Drainage Bag with ENFit® ISO 80369-3 connector **1000ml**
- DBP-1500** PVC Drainage Bag with ENFit® ISO 80369-3 connector **1500ml**
- DBP-2000** PVC Drainage Bag with ENFit® ISO 80369-3 connector **2000ml**

PVC ENTERAL DRAINAGE BAGS WITH HOOKS

- DBP-250HT** PVC Drainage Bag with 50cm inlet tube, ENFit® ISO 80369-3 connector, and hooks **250ml**
- DBP-500HT** PVC Drainage Bag with 50cm inlet tube, ENFit® ISO 80369-3 connector, and hooks **500ml**
- DBP-1000HT** PVC Drainage Bag with 50cm inlet tube, ENFit® ISO 80369-3 connector, and hooks **1000ml**

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

Enteral UK is a trading name of **GBUK Enteral Ltd.**

Rx Only Caution: Federal (U.S.A.) law restricts this device to sale by or on the order of a physician.

Symbol glossary available at www.gbukenteral.com.
Not made with natural rubber latex.

GBUK Group Ltd.

Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0) 1757 288 587
US: 18665 289 422
info@gbukhealthcare.com

QualRep Services BV

Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

Swiss AR Services GmbH

Industriestrasse 47,
6300 Zug,
Switzerland