**GBUKBanana**  
Part of the GBUK Group ●●●●

# Heavy Duty Turning Device

**INSTRUCTIONS FOR USE**

	<b>Page</b>
<b>EN</b> GBUK Banana Heavy Duty Turning Device	4
<b>FR</b> Dispositif de retournement GBUK Banana pour usage intensif	5
<b>IT</b> Dispositivo per rotazione GBUK Banana Heavy Duty	6
<b>ES</b> Producto sanitario de alta resistencia para girar pacientes GBUK Banana	7
<b>DE</b> Strapazierfähige GBUK Banana-Drehhilfe	8
<b>DA</b> GBUK Banana kraftig anordning til at vende patienten	9
<b>NO</b> GBUK Banana kraftig dreieenhet	10
<b>SE</b> GBUK Banana Kraftiga Vändanordning	11
<b>NL</b> GBUK Banana Heavy Duty-draaitoestel	12
<b>PL</b> Wysokowytrzymałe urządzenie do obracania pacjenta GBUK Banana	13
<b>PT</b> Dispositivo de uso intensivo para virar doentes GBUK Banana	14
<b>EL</b> Μηχάνημα περιστροφής ασθενών βαρέως τύπου GBUK Banana	15
<b>TR</b> GBUK Banana Ağır Yük Döndürme Cihazı	16
<b>CS</b> GBUK Banana – pomůcka k otáčení robustnějších pacientů	17
<b>ET</b> GBUK Banana raskete patsientide pööramisseade	18
<b>HU</b> GBUK Banana nagy teherbírású betegforgató eszköz	19
<b>LV</b> GBUK Banana lielai slodzei paredzēta pagriešanas ierīce	20
<b>MT</b> Apparat tat-Tidwir Heavy Duty tal-GBUK Banana	21
<b>RO</b> Dispozitiv de rotire pentru sarcini grele GBUK Banana	22
<b>SL</b> GBUK Banana naprava za obračanje v težki izvedbi	23

**GBUK Group Ltd.**  
Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK  
EU: +44 (0)1757 288 587  
US: 18665 289 422  
www.gbukbanana.com  
info@gbukbanana.com

**QualRep Services BV**  
Utrechtseweg 310 B42  
6812 AR Arnhem,  
The Netherlands

**EN**

## GBUK Banana Heavy Duty Turning Device

### Intended use

GBUK Banana Heavy Duty Turning Devices are intended to be placed on the floor to assist a patient in turning their lower body during a seated transfer.

### Indications for use

Indicated for seated transfer of patients within the patient target group.

### Description

360-degree swivel pad  
Safe Working Load (SWL) limit 225kg

### Patient target group

Suitable for patients who can independently sit up but who require assistance in turning their lower body during a seated transfer.

### Intended users

The device is intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

### Contra-indications

Do not use in direct contact with broken skin.  
Do not use with patients who cannot independently sit up.  
Do not use with patients who exceed the SWL limit.

### Warnings and precautions

- Follow local manual handling policy from your hospital or Royal College of Nursing (RCN) guidelines.
- Do not exceed the 225kg SWL limit.

A risk assessment and visual inspection must be performed by competent person(s) before using the device. This should consider:

- The physical and medical status of the patient, including skin integrity.
- The status of the device. Do not use the device if it appears damaged, wet, or worn.
- The capability of the user.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

### Operating instructions

- Position the device on the floor and place the patient's feet completely on the device, following your local manual handling policy.
- Complete the seated transfer using your chosen seated board. GBUK Banana recommend using a GBUK Banana Seated Board.
- Once the transfer is complete, remove the device.

### Storage and handling

Keep dry and out of direct sunlight.

### Disposal procedures

Follow local guidelines.

### Cleaning

If the device is soiled, thoroughly clean it as soon as possible using alcohol-based cleaning solutions (up to 70%), or normal disinfectant solutions. Manually wipe down the device until the device is visibly clean. If the device does not appear visibly clean, repeat the cleaning step. If the device remains contaminated upon visual inspection following multiple cleaning attempts, discard the device in line with the disposal procedure.

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

**GBUK Group Ltd.**

Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK

EU: +44 (0)1757 288 587

US: 18665 289 422

[www.gbukbanana.com](http://www.gbukbanana.com)

[info@gbukbanana.com](mailto:info@gbukbanana.com)

Symbol glossary available at [www.gbukbanana.com](http://www.gbukbanana.com).

Turning-Device-03 2211 02