Slide Sheets

Washable

INSTRUCTIONS FOR USE

	Page
EN GBUK Banana Washable Slide Sheets	4
FR Draps de repositionnement lavables GBUK Banana	5
IT Teli di posizionamento lavabili di GBUK Banana	6
ES Sábanas de transferencia lavables GBUK Banana	7
DE Waschbare GBUK Banana-Gleitmatten	8
DA GBUK Banana vaskbare glidelagner	9
NO GBUK Banana glidelaken som kan vaskes	10
SV GBUK Banana tvättbara glidlakan	11
NL GBUK Banana Wasbare Glijlakens	12
PL Maty ślizgowe z możliwością prania GBUK	13
PT Lençóis de transferência laváveis GBUK Banana	14
EL Πλενόμενα σεντόνια ολίσθησης GBUK Banana	15
TR GBUK Banana Yıkanabilir Hasta Kaydırma Kumaşları	16
CS Omyvatelné kluzné podložky GBUK Banana	17
ET GBUK Banana pestavad libistuslinad	18
HU A GBUK Banana Mosható Tolólapok	19
LV GBUK Banana mazgājамie slīdes palagi	20
MT Lożor għall-Ġarr u li Jistgħu Jinħaslu tal-GBUK Banana	21
RO Foile glisante lavabile pentru transferul pacienților GBUK Banana	22
SL Pralne drsne blazine GBUK Banana,	23

GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

EC REP **QualRep Services BV**
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

CH REP **Swiss AR Services AG**
Industriestrasse 47
6300 Zug, Switzerland

EN

GBUK Banana Washable Slide Sheets, including:

Washable Versal Slide Sheet
Washable Hemmed Flat Sheets
Washable Hemmed Flat Sheets with Handles
Washable Hemmed Tubular Sheets

Instructions for Use

Please read these instructions before use.

Intended use

GBUK Banana Washable Slide Sheets are intended to reduce friction and shear during the task of moving, repositioning, or turning a patient on a surface.

Indications for use

These devices are indicated for use on patients who require assistance to be moved, repositioned, or turned.

Description

GBUK Banana Washable Slide Sheets are available as flat sheets and tubular sheets. Flat sheets are available with or without handles.

Patient target group

This device is suitable for use on patients who require assistance to be moved, repositioned, or turned.

Intended users

GBUK Banana Washable Slide Sheets are intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

Contra-indications

Do not use in direct contact with broken skin.
If a safe working load (SWL) limit is specified, do not use with patients who exceed the SWL limit.

Warnings and precautions

- Follow local manual handling policy from your hospital or Royal College of Nursing guidelines.
- Do not use the slide sheets to lift a patient.
- Do not exceed the 225kg SWL limit when using GBUK Banana Washable Slide Sheets with handles

A risk assessment and visual inspection must be performed by competent person(s) before using the Slide Sheet. This should consider:

- The physical and medical status of the patient, including skin integrity.
 - The status of the device. The material must be intact and not exhibit any splits or tears. If there are handles on the device, the stitching should be inspected to ensure the stitching is intact. Do not use the device if it appears damaged, wet, or worn.
 - The suitability of the slide sheet, including the size.
- In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

Operating instructions

- Follow your local manual handling policy to position the slide sheet beneath the patient.
- GBUK Banana recommends using two layers of slide sheet material for effective manoeuvres.
- GBUK Banana recommends the use of GBUK Banana Lateral Transfer Boards to complete a lateral transfer with slide sheets.
- Once the device is in position follow your local manual handling policy to move, laterally transfer, reposition or turn a patient.
- Laundry the device in line with the cleaning instructions if the device is soiled, or before reassigning to a new patient.

Storage and handling

Keep dry and out of direct sunlight.

Disposal procedures

Dispose of the device if device performance or integrity is compromised. Follow your hospital protocol.

Cleaning

Suitable for laundering. As soon as possible, machine wash at 70°C. Tumble dry on a low heat or leave to air dry. If the device remains contaminated upon visual inspection following multiple cleaning attempts, discard the device in line with hospital protocol. Do not bleach, dry clean or iron.

Not made with natural rubber latex.

Symbol glossary available at www.gbukbanana.com

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Ni izdelano iz naravnega lateksa.

Slovar simbolov najdete na naslovu www.gbukbanana.com

GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

Symbol glossary available at www.gbukbanana.com
Not made with natural rubber latex.

Caution: Federal law restricts this device to sale by or on the order of a manual handler, trained healthcare professional and others appropriately trained in use of the device.

GBUK Group Ltd.

Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

Swiss AR Services AG
Industriestrasse 47
6300 Zug, Switzerland