**GBUKBanana**  
Part of the GBUK Group ●●●●

# Lateral Transfer Boards

**INSTRUCTIONS FOR USE**

**GBUK Group Ltd.**  
Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK  
**EU: +44 (0)1757 288 587**  
**US: 18665 289 422**  
[www.gbukbanana.com](http://www.gbukbanana.com)  
[info@gbukbanana.com](mailto:info@gbukbanana.com)

**EC REP** **QualRep Services BV**  
Utrechtseweg 310 B42  
6812 AR Arnhem,  
The Netherlands

**CH REP** **Swiss AR Services AG**  
Industriestrasse 47,  
6300 Zug,  
Switzerland

	<b>Page</b>
<b>EN</b> GBUK Banana Lateral Transfer Board	4
<b>FR</b> Planche de transfert latéral GBUK Banana	5
<b>IT</b> Asse per il trasferimento laterale di GBUK Banana	6
<b>ES</b> Tabla para traslado lateral GBUK Banana	7
<b>DE</b> GBUK Banana-Brett zur seitlichen Umlagerung	8
<b>DA</b> GBUK Banana lateral forflytningsplade	9
<b>NO</b> GBUK Banana lateralt overføringsbrett	10
<b>SV</b> GBUK Banana lateral förflyttningsbräda	11
<b>NL</b> GBUK Banana Laterale Transferplank	12
<b>PL</b> Deska do transferu boczno GBUK Banana	13
<b>PT</b> Tábua de transferência lateral GBUK Banana	14
<b>EL</b> Πλάκα Πλευρικής Μεταφοράς GBUK Banana	15
<b>TR</b> GBUK Banana Lateral Hasta Transfer Tahtası	16
<b>CS</b> Deska pro boční přenos GBUK Banana	17
<b>ET</b> GBUK Banana lateraalne tõstelaud	18
<b>HU</b> GBUK Banana Oldal Irányú Áthelyező Lap	19
<b>LV</b> GBUK Banana dēlis pārvietošanai uz sāniem	20
<b>MT</b> Bordijiet għat-Trasferiment Laterali tal-GBUK Banana	21
<b>RO</b> Placă pentru transfer lateral GBUK Banana	22
<b>SL</b> Deska za bočno prestavljanje GBUK Banana	23

EN

## GBUK Banana Lateral Transfer Board

### Instructions for Use

Please read these instructions before use.

### Intended use

GBUK Banana Lateral Transfer Boards are intended to assist in the lateral transfer of patients across a gap between two surfaces.

### Indications for use

These devices are indicated for use on patients who require assistance in completing a lateral transfer.

### Description

Lightweight, high strength lateral transfer boards which are available as rigid or folding types.

Product Code	Product Description
QP-059150-R	Banana Lateral Rigid Board 59cm x 150cm
QP-059180-R	Banana Lateral Rigid Board 59cm x 180cm
QP-074150-R	Banana Lateral Rigid Board 74cm x 150cm
QP-074180-R	Banana Lateral Rigid Board 74cm x 180cm
QP-059150-F	Banana Lateral Folding Board 59cm x 150cm*
QP-059180-F	Banana Lateral Folding Board 59cm x 180cm*
QP-074150-F	Banana Lateral Folding Board 74cm x 150cm*
QP-074180-F	Banana Lateral Folding Board 74cm x 180cm*

\*The folding type enables transfer of patients in a semi-recumbent position.

### Patient target group

This device is suitable for patients who require assistance in completing a lateral transfer.

### Intended users

GBUK Banana Lateral Transfer Boards are intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

### Contra-indications

Do not use in direct contact with broken skin.

### Warnings and precautions

- Follow local manual handling policy from your hospital or Royal College of Nursing guidelines.
- Do not use the boards for lifting.
- Do not use the boards to bridge a gap greater than 15 cm.

A risk assessment and visual inspection must be performed by competent person(s) before using the Lateral Transfer Board. This should consider:

- The physical and medical status of the patient, including skin integrity.
- The status of the device. The anti-slip strips must be intact and undamaged. Do not use the device if it appears damaged, wet, or worn.
- The suitability of the board, including the type of rigid or folding board and size.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

### Directions for use

- Ensure that both transfer surfaces are stable. If either is a mobile device, confirm that wheels/casters are locked and/or any other anchoring devices are engaged.
- Follow your local manual handling policy to position the board beneath the patient, and across the gap between the first and second transfer surfaces. The gap must be no greater than 15 cm.
- Check that the anti-slip strips are in contact with both transfer surfaces.
- To reduce the effort required to conduct the transfer, the use of GBUK Banana transfer or slide sheets is recommended. Refer to specific device IFUs and local manual handling guidelines to position the transfer or slide sheet beneath the patient.
- Once the board is in position, laterally transfer the patient across the two surfaces.
- After the manoeuvre is complete, remove the board and clean as per instructions below.

### Cleaning

Thoroughly clean the device as soon as possible using alcohol-based cleaning solutions (up to 70%) or normal disinfectant solutions. Manually wipe down the device until the device is visibly clean. If the device does not appear visibly clean, repeat the cleaning step. If the device remains contaminated upon visual inspection following multiple cleaning attempts, discard of the device in line with local hospital protocol.

### Storage and handling

GBUK Banana Lateral Transfer Boards can be stored using any of the handholds on the board. They can be stored flat or folded in half. Folding Lateral Transfer Boards should be folded with the anti-slip strips on the outside. Do not fold the board with the anti-slip strips on the inside as this can damage the board. Keep dry and out of direct sunlight.

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

Symbol glossary available at [www.gbukbanana.com](http://www.gbukbanana.com)  
Not made with natural rubber latex.

**GBUK Group Ltd.**

Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK  
**EU: +44 (0)1757 288 587**  
**US: 18665 289 422**  
[www.gbukbanana.com](http://www.gbukbanana.com)  
[info@gbukbanana.com](mailto:info@gbukbanana.com)

**QualRep Services BV**  
Utrechtseweg 310 B42  
6812 AR Arnhem,  
The Netherlands

**Swiss AR Services AG**  
Industriestrasse 47  
6300 Zug, Switzerland