**GBUKBanana**  
Part of the GBUK Group ●●●●

# Seated Boards

**INSTRUCTIONS FOR USE**

	<b>Page</b>
<b>EN</b> GBUK Banana Seated Boards	4
<b>FR</b> Planches de transfert assis GBUK Banana	5
<b>IT</b> Sedili di trasferimento di GBUK Banana	6
<b>ES</b> Tabla curva de transferencia GBUK Banana	7
<b>DE</b> GBUK Banana-Sitzbrett	8
<b>DA</b> GBUK Banana siddeplader	9
<b>NO</b> GBUK Banana brett med sete	10
<b>SE</b> GBUK Banana sittbrädor	11
<b>NL</b> GBUK Banana Zitplanken	12
<b>PL</b> Deska do transferu pacjenta w pozycji siedzącej GBUK Banana	13
<b>PT</b> Tábua Assento GBUK Banana	14
<b>EL</b> Πλάκες για Καθιστή θέση GBUK Banana	15
<b>TR</b> GBUK Banana Oturmalı Transfer Tahtaları	16
<b>CS</b> Desky pro sezení GBUK Banana	17
<b>ET</b> GBUK Banana istelauad	18
<b>HU</b> GBUK Banana Ülő lapok	19
<b>LV</b> GBUK Banana dēlis sēdus pozīcijai	20
<b>MT</b> Bordijiet Bilqiegħda tal-GBUK Banana	21
<b>RO</b> Plăci pentru șezut GBUK Banana	22
<b>SL</b> Sedežne deske GBUK Banana	23

**GBUK Group Ltd.**  
Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK  
EU: +44 (0)1757 288 587  
US: 18665 289 422  
www.gbukbanana.com  
info@gbukbanana.com

**QualRep Services BV**  
Utrechtseweg 310 B42  
6812 AR Arnhem,  
The Netherlands

EN

## GBUK Banana Seated Boards

### Instructions for Use

Please read these instructions before use.

### INTENDED USE

GBUK Banana Seated Boards are intended to assist in the seated transfer of a patient from one surface to another.

### INDICATIONS FOR USE

Indicated for seated lateral transfer of patients within the patient target group stated below.

### DESCRIPTION

Lightweight and curved seated boards for seated lateral transfers.

Product Code	Product Description	Safe Working Load (SWL)
QBB01	Original Seated Board 77cm x 35cm	150 kg
BB-Prime	BB Prime Seated Board 26cm x 62cm	200 kg
Q2Board-Q2s	Q2 Seated Board 33.5cm x 77cm	285 kg

### PATIENT TARGET GROUP

Suitable for patients who can independently sit up but require assistance for a seated lateral transfer.

### INTENDED USERS

GBUK Banana Seated Boards are intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

### CONTRA-INDICATIONS

Do not use in direct contact with broken skin.  
Do not use with patients who cannot independently sit up.  
Do not use with patients who are disorientated, off-balance, sleepy, or dizzy.  
Do not use with patients who exceed the SWL limit.

### WARNINGS AND PRECAUTIONS

- Follow local manual handling policy from your hospital or Royal College of Nursing (RCN) guidelines.
- Do not use the boards for lifting.
- The anti-slip pads MUST be in contact with both transfer surfaces before commencing the transfer.

A risk assessment and visual inspection must be performed by competent person(s) before using the Seated Board. This should consider:

- The physical and medical status of the patient, including skin integrity.
- The status of the device. The anti-slip pads must be dry, intact, and undamaged. Do not use the device if it appears damaged, wet, or worn.
- The suitability of the Seated Board including the correct size and SWL limit.
- The capability of the user.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

### DIRECTIONS FOR USE

- Ensure both transfer surfaces are stable. If either is a mobile device, confirm that wheels/castors are locked and/or any anchoring devices are engaged.
- Place the board beneath one leg. One anti-slip pad must be completely in contact with the first transfer surface.
- Place the other end of the board onto the second transfer surface, ensuring the anti-slip pad is completely in contact with the transfer surface. The gap must be no greater than the distance between the two anti-slip pads.
- For a carer-assisted transfer, follow local manual handling guidelines or RCN guidelines.
- Complete the transfer by sliding the patient across the board to a new seated position.
- Transfers can be undertaken with the patient's legs either on the inside or outside curve of the board. GBUK Banana recommends using the inside curve to achieve a more streamlined motion.

### STORAGE AND HANDLING

Keep dry and out of direct sunlight.

### DISPOSAL PROCEDURES

Follow local guidelines.

### CLEANING

If the device is soiled, thoroughly clean it as soon as possible using alcohol-based cleaning solutions (up to 70%), or normal disinfectant solutions. Manually wipe down the device until the device is visibly clean. If the device does not appear visibly clean, repeat the cleaning step. If the device remains contaminated upon visual inspection following multiple cleaning attempts, discard of the device in line with the disposal procedure.

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

**GBUK Group Ltd.**

Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK

EU: +44 (0)1757 288 587

US: 18665 289 422

[www.gbukbanana.com](http://www.gbukbanana.com)

[info@gbukbanana.com](mailto:info@gbukbanana.com)

Symbol glossary available at [www.gbukbanana.com](http://www.gbukbanana.com).

SeatedBoards-03 2303 03