GBUKBanana
Part of the GBUK Group ●●●●

Slide Sheets

Disposable

INSTRUCTIONS FOR USE

	Page
EN GBUK Banana Disposable Slide Sheets	4
FR Draps de repositionnement jetables GBUK Banana	5
IT Teli di posizionamento monouso di GBUK Banana	6
ES Sábana de transferencia desechable GBUK Banana	7
DE GBUK Banana-Einweggleitmatten	8
DA GBUK Banana engangsglidelagner	9
NO GBUK Banana engangs glidelaken	10
SE GBUK Banana glidlakan för engångsbruk	11
NL GBUK Banana Glijlakens voor Eénmalig Gebruik	12
PL Jednorazowe maty ślizgowe GBUK Banana	13
PT Lençóis de transferência descartáveis GBUK Banana	14
EL Σεντόνια Ολίσθησης μίας χρήσης GBUK Banana	15
TR Σεντόνια Ολίσθησης μίας χρήσης GBUK Banana	16
CS Jednorázové kluzné podložky GBUK Banana	17
ET GBUK Banana ühekordsed libistuslinad	18
HU GBUK Banana Egyszer Használatos Tolólapok	19
LV GBUK Banana vienreizlietojamie slīdes palagi	20
MT Lożor għall-Ġarr Mhux Riutilizzabbli tal-GBUK Banana	21
RO Foi glisante consumabile pentru transferul pacienților GBUK Banana	22
SL Drsne blazine GBUK Banana za enkratno uporabo	23

GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

EN

GBUK Banana Disposable Slide Sheets

Instructions for Use

Please read these instructions before use.

INTENDED USE

GBUK Banana Disposable Slide Sheets reduce friction and shear during the task of moving, repositioning, or turning a patient on a surface.

INDICATIONS FOR USE

These devices are indicated for use on patients who require assistance to be moved, repositioned, or turned.

DESCRIPTION

GBUK Banana Disposable Slide Sheets are available on reels as flat sheets and tubular sheets in various sizes.

PATIENT TARGET GROUP

This device is suitable for use on patients who require assistance to be moved, repositioned, or turned.

INTENDED USERS

GBUK Banana Disposable Slide Sheets are intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

CONTRA-INDICATIONS

Do not use in direct contact with broken skin.

WARNINGS AND PRECAUTIONS

- Follow local manual handling policy from your hospital or Royal College of Nursing guidelines.
- Do not use slide sheets to lift a patient.
- For single patient use only. Cross patient use of this device poses a risk of infection and/or cross contamination.

A risk assessment and visual inspection must be performed by competent person(s) before using the Slide Sheet. This should consider:

- The physical and medical status of the patient, including skin integrity.
- The status of the device. The device must be intact and not exhibit any splits or tears. Do not use the device if it appears damaged, wet, or worn.
- The suitability of the slide sheet, including the size.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

DIRECTIONS FOR USE

- Feed the first slide sheet of the reel through the slot in the dispenser.
- Pull the slide sheet out of the dispenser until the perforation is visible. Gently tear along the perforation until the slide sheet is detached from the reel.
- For tubular slide sheets, pull open both sides before use.
- For flat slide sheets, use two layers and ensure the yellow surfaces are facing each other.
- Follow your local manual handling policy to position the slide sheets beneath the patient.
- GBUK Banana recommends the use of GBUK Banana Lateral Transfer Boards to complete a lateral transfer with slide sheets.
- Once the device is in position follow your local manual handling policy to move, laterally transfer, reposition or turn a patient.

STORAGE AND HANDLING

Keep dry and out of direct sunlight.

DISPOSAL PROCEDURES

Follow local guidelines.

CLEANING

GBUK Banana Disposable Slide Sheets are designed to be disposable and single patient use only. If lightly soiled, manually wipe down the slide sheet as soon as possible using alcohol-based cleaning solutions (up to 70%) or normal disinfectant solutions until the slide sheet is visibly clean. If the slide sheet does not appear visibly clean and remains contaminated upon visual inspection, discard the slide sheet in line with local hospital protocol.

Not made with natural rubber latex.

Symbol glossary available at www.gbukbanana.com

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Ni izdelano iz naravnega lateksa.

Slovar simbolov najdete na naslovu www.gbukbanana.com

GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

GBUK Group Ltd.

Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK

EU: +44 (0)1757 288 587

US: 18665 289 422

www.gbukbanana.com

info@gbukbanana.com

Symbol glossary available at www.gbukbanana.com.

SlideSheetsDisposable-03 2210 02