GBUKBanana
Part of the GBUK Group ●●●●

Slide Sheets

Single Patient Use

INSTRUCTIONS FOR USE

 **GBUK Group Ltd.**
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

EC REP **QualRep Services BV**
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

CH REP **Swiss AR Services AG**
Industriestrasse 47
6300 Zug, Switzerland

	Page
EN GBUK Banana Single Patient Use (SPU) Slide Sheets	4
FR Draps de repositionnement à l'usage d'un seul patient GBUK Banana	5
IT Teli di posizionamento monouso di GBUK Banana	6
ES Sábanas de transferencia GBUK Banana para uso en un solo paciente	7
DE GBUK Banana-Gleitmatten zum mehrmaligen Gebrauch an einem einzelnen Patienten (SPU)	8
DA GBUK Banana glidelagner til brug på én patient (SPU: Single Patient Use)	9
NO GBUK Banana glidelaken til engangsbruk (SPU)	10
SV GBUK Banana glidlakan för enpatientsbruk (SPU)	11
NL GBUK Banana Glijlakens voor Gebruik door Eén Patiënt (SPU)	12
PL Maty ślizgowe jednopacjentowe GBUK Banana	13
PT Lençóis de transferência, de utilização num único paciente GBUK Banana (SPU)	14
EL Σεντόνια Ολίσθησης για χρήση σε έναν ασθενή μόνο (SPU) GBUK Banana	15
TR GBUK Banana Tek Hastada Kullanım İçin (SPU) Kaydırma Kumaşları	16
CS Kluzné podložky GBUK Banana pro jednoho pacienta (SPU)	17
ET GBUK Banana libistuslinad ühel patsiendil (SPU) kasutamiseks	18
HU GBUK Banana Egy Beteg Általi Használatú (SPU) Tolólapok	19
LV GBUK Banana vienam pacientam paredzēti (SPU) slīdes palagi	20
MT Lożor għall-Ġarr b'Użu minn Pazjent Wieħed (SPU) tal-GBUK Banana	21
RO Foi glisante de unică folosință (SPU) pentru transferul pacienților GBUK Banana	22
SL Drsne blazine GBUK Banana, uporabne za enega pacienta (SPU)	23

EN

GBUK Banana Single Patient Use (SPU) Slide Sheets, including:

SPU Flat Slide Sheets
SPU Flat Slide Sheets with Handles
SPU PATPAQ Slide Sheets
SPU Tubular Slide Sheets
SPU Versal Slide Sheets
SPU Two Toned Tubular Slide Sheets
SPU ECO Flat Slide Sheets
SPU ECO Flat Slide Sheets with Handles
SPU ECO Tubular Slide Sheets

Instructions For Use

Please read these instructions before use.

Intended use

GBUK Banana Single Patient Use (SPU) Slide Sheets are intended to reduce friction and shear during the task of moving, repositioning, or turning a patient on a surface.

Indications for use

These devices are indicated for use on patients who require assistance to be moved, repositioned, or turned.

Description

GBUK Banana SPU Slide Sheets are available as flat sheets and tubular sheets in various sizes. Flat sheets are available with and without handles.

Patient target group

This device is suitable for use on patients who require assistance to be moved, repositioned, or turned.

Intended users

GBUK Banana SPU Slide Sheets are intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

Contra-indications

Do not use in direct contact with broken skin. If a safe working load (SWL) limit is specified, do not use with patients who exceed the SWL limit.

Warnings and precautions

- Follow local manual handling policy from your hospital or Royal College of Nursing guidelines.
- Do not use the slide sheets to lift a patient.
- Do not launder or bleach the device.
- For single patient use only. Cross patient use of this device poses a risk of infection and/or cross contamination.
- Do not exceed the 225kg Safe Working Load (SWL) limit when using GBUK Banana SPU Slide Sheets with handles.

A risk assessment and visual inspection must be performed by competent person(s) before using the Slide Sheet.

This should consider:

- The physical and medical status of the patient, including skin integrity.
- The status of the device. The material must be intact and not exhibit any splits or tears. If there are handles on the device, the stitching should be inspected to ensure the stitching is intact. Do not use the device if it appears damaged, wet, or worn.
- The suitability of the slide sheet, including the size.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

Operating instructions

- Follow your local manual handling policy to position the slide sheet beneath the patient.
- GBUK Banana recommends using two layers of slide sheet material for effective manoeuvres.
- GBUK Banana recommends the use of GBUK Banana Lateral Transfer Boards to complete a lateral transfer with slide sheets.
- Once the device is in position follow your local manual handling policy to move, laterally transfer, reposition, or turn a patient.

Storage and handling

Keep dry and out of direct sunlight.

Disposal procedures

Follow local guidelines.

Cleaning

GBUK Banana SPU Slide Sheets are designed to be reused with the same patient i.e., the device is single patient use only. If lightly soiled, manually wipe down the slide sheet as soon as possible using alcohol-based cleaning solutions (up to 70%) or normal disinfectant solutions until the slide sheet is visibly clean. If the slide sheet does not appear visibly clean and remains contaminated upon visual inspection, discard the slide sheet in line with local hospital protocol.

Not made with natural rubber latex.

Symbol glossary available at www.gbukbanana.com

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Ni izdelano iz naravnega lateksa.

Slovar simbolov najdete na naslovu www.gbukbanana.com

GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

Symbol glossary available at www.gbukbanana.com
Not made with natural rubber latex.

Caution: Federal law restricts this device to sale by or on the order of a manual handler, trained healthcare professional and others appropriately trained in use of the device.

GBUK Group Ltd.

Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

Swiss AR Services AG
Industriestrasse 47
6300 Zug, Switzerland