**GBUKBanana**  
Part of the GBUK Group ●●●●

# Transfer Sheets

Single Patient Use

**INSTRUCTIONS FOR USE**

	<b>Page</b>
<b>EN</b> GBUK Banana Transfer Sheets	4
<b>FR</b> Draps de transfert GBUK Banana	5
<b>IT</b> Teli di trasferimento di GBUK Banana	6
<b>ES</b> Sábanas de traslado GBUK Banana	7
<b>DE</b> GBUK Banana-Umlagerungstücher	8
<b>DA</b> GBUK Banana forflytningslagner	9
<b>NO</b> GBUK Banana overføringslaken	10
<b>SE</b> GBUK Banana förflyttningsmattor	11
<b>NL</b> GBUK Banana Transferlakens	12
<b>PL</b> Maty do transferu pacjenta GBUK Banana	13
<b>PT</b> Lençóis de transferência GBUK Banana	14
<b>EL</b> Σεντόνια Μεταφοράς GBUK Banana	15
<b>TR</b> GBUK Banana Hasta Transfer Kumaşları	16
<b>CS</b> Podložky pro přenos GBUK Banana	17
<b>ET</b> GBUK Banana teisalduslinad	18
<b>HU</b> GBUK Banana Áthelyező Lapok	19
<b>LV</b> GBUK Banana pārvietošanas palagi	20
<b>MT</b> Lożor għat-Trasferiment tal-GBUK Banana	21
<b>RO</b> Foi pentru transferul pacienților GBUK Banana	22
<b>SL</b> Blazine za prenašanje GBUK Banana	23

**GBUK Group Ltd.**  
Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK  
EU: +44 (0)1757 288 587  
US: 18665 289 422  
www.gbukbanana.com  
info@gbukbanana.com

**QualRep Services BV**  
Utrechtseweg 310 B42  
6812 AR Arnhem,  
The Netherlands

EN

## GBUK Banana Transfer Sheets

### Instructions for Use

Please read these instructions before use.

### INTENDED USE

GBUK Banana Transfer Sheets are intended to assist in the transfer of patients between surfaces.

### INDICATIONS FOR USE

These devices are indicated for use on patients who require assistance to be laterally transferred.

### DESCRIPTION

GBUK Banana Transfer Sheets are made from a non-woven material with six ergonomically placed handles on each side.

### PATIENT TARGET GROUP

This device is suitable for use on patients who require assistance to be laterally transferred.

### INTENDED USERS

GBUK Banana Transfer Sheets are intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

### CONTRA-INDICATIONS

Do not use in direct contact with broken skin. Do not use with a patient who exceeds the Safe Working Load (SWL) limit

### WARNINGS AND PRECAUTIONS

- Follow local manual handling policy from your hospital or Royal College of Nursing guidelines.
- Do not use the transfer sheets to lift a patient.
- Do not exceed the 500kg Safe Working Load (SWL) when using GBUK Banana Transfer Sheets.
- Do not leave under a patient on an inclined/declined surface.
- For single patient use only. Cross patient use of this device poses a risk of infection and/or cross contamination.
- Do not launder or bleach the device.

A risk assessment and visual inspection must be performed by competent person(s) before using the Transfer Sheet. This should consider:

- The physical and medical status of the patient, including skin integrity.
- The status of the device. The material must be intact and not exhibit any splits or tears. Do not use the device if it appears damaged or worn.
- The suitability of the transfer sheet, including the size.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

### DIRECTIONS FOR USE

- Follow your local manual handling policy to position the transfer sheet beneath the patient.
- GBUK Banana recommends the use of GBUK Banana Lateral Transfer Boards to complete a lateral transfer with transfer sheets.
- GBUK Banana recommends the use of GBUK Banana Transfer Handles to increase the handler's reach.
- Once the device is in position, follow your local manual handling policy to laterally transfer the patient.

### STORAGE AND HANDLING

Keep dry and out of direct sunlight.

### DISPOSAL PROCEDURES

Follow your hospital protocol.

### CLEANING

GBUK Banana Transfer Sheets are intended for single patient use. If lightly soiled, manually wipe down the transfer sheet as soon as possible using alcohol-based cleaning solutions (up to 70%) or normal disinfectant solutions until the transfer sheet is visibly clean. If the transfer sheet does not appear visibly clean and remains contaminated upon visual inspection, discard the transfer sheet in line with local hospital protocol.

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

**GBUK Group Ltd.**

Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK

EU: +44 (0)1757 288 587

US: 18665 289 422

[www.gbukbanana.com](http://www.gbukbanana.com)

[info@gbukbanana.com](mailto:info@gbukbanana.com)