GBUKBanana
Part of the GBUK Group ●●●●

Transfer Sheets

Single Patient Use

INSTRUCTIONS FOR USE

	Page
EN GBUK Banana Transfer Sheets	4
FR Draps de transfert GBUK Banana	5
IT Teli di trasferimento di GBUK Banana	6
ES Sábanas de traslado GBUK Banana	7
DE GBUK Banana-Umlagerungstücher	8
DA GBUK Banana forflytningslagner	9
NO GBUK Banana overføringslaken	10
SV GBUK Banana förflyttningsmattor	11
NL GBUK Banana Transferlakens	12
PL Maty do transferu pacjenta GBUK Banana	13
PT Lençóis de transferência GBUK Banana	14
EL Σεντόνια Μεταφοράς GBUK Banana	15
TR GBUK Banana Hasta Transfer Kumaşları	16
CS Podložky pro přenos GBUK Banana	17
ET GBUK Banana teišalduslinad	18
HU GBUK Banana Áthelyező Lapok	19
LV GBUK Banana pārvietošanas palagi	20
MT Lożor għat-Trasferiment tal-GBUK Banana	21
RO Foi pentru transferul pacienților GBUK Banana	22
SL Blazine za prenašanje GBUK Banana	23

GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

EC REP **QualRep Services BV**
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

CH REP **Swiss AR Services AG**
Industriestrasse 47
6300 Zug, Switzerland

EN

GBUK Banana Transfer Sheets

Instructions For Use

Please read these instructions before use.

Intended use

GBUK Banana Transfer Sheets are intended to assist in the transfer of patients between surfaces.

Indications for use

These devices are indicated for use on patients who require assistance to be laterally transferred.

Description

GBUK Banana Transfer Sheets are made from a non-woven material with six ergonomically placed handles on each side.

Patient target group

This device is suitable for use on patients who require assistance to be laterally transferred.

Intended users

GBUK Banana Transfer Sheets are intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

Contra-indications

Do not use in direct contact with broken skin.
Do not use with a patient who exceeds the Safe Working Load (SWL) limit

Warnings and precautions

- Follow local manual handling policy from your hospital or Royal College of Nursing guidelines.
- Do not use the transfer sheets to lift a patient.
- Do not exceed the 500kg Safe Working Load (SWL) when using GBUK Banana Transfer Sheets.
- Do not leave under a patient on an inclined/declined surface.
- For single patient use only. Cross patient use of this device poses a risk of infection and/or cross contamination.
- Do not launder or bleach the device.

A risk assessment and visual inspection must be performed by competent person(s) before using the Transfer Sheet. This should consider:

- The physical and medical status of the patient, including skin integrity.
- The status of the device. The material must be intact and not exhibit any splits or tears. Do not use the device if it appears damaged or worn.
- The suitability of the transfer sheet, including the size.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

Operating instructions

- Follow your local manual handling policy to position the transfer sheet beneath the patient.
- GBUK Banana recommends the use of GBUK Banana Lateral Transfer Boards to complete a lateral transfer with transfer sheets.
- GBUK Banana recommends the use of GBUK Banana Transfer Handles to increase the handler's reach.
- Once the device is in position, follow your local manual handling policy to laterally transfer the patient.

Storage and handling

Keep dry and out of direct sunlight.

Disposal procedures

Follow your hospital protocol.

Cleaning

GBUK Banana Transfer Sheets are intended for single patient use. If lightly soiled, manually wipe down the transfer sheet as soon as possible using alcohol-based cleaning solutions (up to 70%) or normal disinfectant solutions until the transfer sheet is visibly clean. If the transfer sheet does not appear visibly clean and remains contaminated upon visual inspection, discard the transfer sheet in line with local hospital protocol.

Not made with natural rubber latex.

Symbol glossary available at www.gbukbanana.com

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Ni izdelano iz naravnega lateksa.

Slovar simbolov najdete na naslovu www.gbukbanana.com

GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

Symbol glossary available at www.gbukbanana.com
Not made with natural rubber latex.

Caution: Federal law restricts this device to sale by or on the order of a manual handler, trained healthcare professional and others appropriately trained in use of the device.

GBUK Group Ltd.

Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

Swiss AR Services AG
Industriestrasse 47
6300 Zug, Switzerland