B-Assist

INSTRUCTIONS FOR USE

	Page
EN B-Assist	4
FR B-Assist	6
IT B-Assist	8
ES B-Assist	10
DE B-Assist	12
DA B-Assist	14
NO B-Assist	16
SE B-Assist	18
NL B-Assist	20
PL B-Assist	22
PT B-Assist	24
EL B-Assist	26
TR B-Assist	28
CS B-Assist	30
ET B-Assist	32
HU B-Assist	34
LV B-Assist	36
MT B-Assist	38
RO B-Assist	40
SL B-Assist	42

GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587
US: 18665 289 422
www.gbukbanana.com
info@gbukbanana.com

QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

EN

B-Assist

INTENDED USE

The Assist is intended to turn patients into a prone position in the same bed or from surface to surface and back again. It may also be used to turn, reposition, assist in the cleaning of patients, and moving legs out of bed.

INDICATIONS FOR USE

Indicated for use on patients who require assistance to be proned, turned, or repositioned.

DESCRIPTION

Unique and easy-to-use one piece device. Staff maintain complete control during the entire procedure. Safe working load (SWL) limit of up to 225kg.

PATIENT TARGET GROUP

This device is suitable for patients who require assistance to be proned, turned, or repositioned.

INTENDED USERS

The Assist is intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

CONTRAINDICATIONS

Do not use in direct contact with broken skin. Do not use with a patient who exceeds the SWL limit.

WARNINGS AND PRECAUTIONS

- Follow local manual handling policy from your hospital or Royal College of Nursing guidelines.
- Do not use the device for lifting.
- For single patient use only. Cross patient use of this device poses a risk of infection and/or cross contamination. Discard the device after 30 days of usage.

Always use the Assist with GBUK Banana Slide Sheets underneath.

A risk assessment and visual inspection must be performed by competent person(s) before using the Assist. This should consider:

- The physical and medical status of the patient, including skin integrity.
- The status of the device. The device must be intact and not exhibit any splits or tears. Do not use the device if it appears damaged, wet, or worn.
- The suitability of the device.
- The capability of the user.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

DIRECTIONS FOR USE

INSERTION USING THE ROLLING METHOD:

- To insert the Assist, roll the patient onto their side and place the Assist over the patient between the scapula and sacral area (shiny surface away from patient).
- Roll the remaining section underneath the patient and insert Slide Sheets (either flat or Versal).
- Roll the patient back and pull the Assist and Slide Sheets through. The Assist should be directly underneath the patient.

INSERTION USING THE FOLDING METHOD:

- This can be achieved with either 2 flat Slide Sheets or a Versal Slide Sheet. Lay the Slide Sheets on a surface and fold over both layers around 15cm in depth. Continue to fold down until you reach the last quarter of the Slide Sheets.
- Insert half of the Assist underneath the Slide Sheets and fold the other half on top of the Slide Sheets and tuck under.
- Turn the Slide Sheets and Assist over and position underneath the patient's head and shoulders. The Assist should be just below the shoulders before starting to unfold.
- Hold the Slide Sheets and Assist in place with your outside hand. Use the inside hand to unfold one fold at a time underneath the patient.
- Both caregivers must unfold each fold at the same time while keeping their hands flat against the surface.
- Once unfolded, the Assist should be directly underneath the patient, positioned between the scapula and sacral area.

FACILITATING A TURN:

- Use the handles on the Assist to move the patient to the edge of the bed and pass the Assist over the top of the patient.
- One caregiver should use the top layer of the Slide Sheet to pull towards them while the other caregiver uses the Assist to turn the patient onto their side.

FACILITATING THE PRONE POSITION ON THE SAME SURFACE:

- This procedure can be completed with 2 caregivers. However, you should complete an individual risk assessment to determine the correct number of caregivers required for this manoeuvre.
- Use the handles on the Assist to move the patient to the edge of the bed and pass the Assist over the top of the patient.
- One caregiver should use the top layer of the Slide Sheet to pull towards them while the other caregiver uses the Assist to turn the patient onto their side.
- Continue to pull the top layer of Slide Sheet while the other caregiver uses the Assist to prone the patient.
- If the patient is not in the middle of the surface and the Slide Sheets are still in position, a lateral or medial movement can be completed in order to achieve the correct position on the surface.
- Remove the Slide Sheets while the patient is in the prone position. You may leave the Assist in position if your risk assessment permits it.

FACILITATING A LATERAL TRANSFER:

- Complete an individual risk assessment to determine the correct number of caregivers required for this manoeuvre.
- Insert the Assist and Slide Sheets using either one of the methods described above.
- Bring the receiving surface into position at a slightly lower height than the starting surface. Add more Slide Sheets to the receiving surface if required.
- Insert the lateral transfer board to bridge the gap between the surfaces.
- Hold the sides of the Assist with your knuckles facing out and pull the Assist until the patient is at the desired position.
- Remove the Assist and Slide Sheets by rolling the patient onto their side.

STORAGE AND HANDLING

Keep dry and out of direct sunlight.

DISPOSAL PROCEDURES

Follow your hospital protocol

CLEANING & DISINFECTION

The Assist is intended for single patient use and must not be laundered. If lightly soiled, manually wipe down the Assist as soon as possible using alcohol-based cleaning solutions (up to 70%) or normal disinfectant solutions until the Assist is visibly clean. If the Assist does not appear visibly clean and remains contaminated upon visual inspection discard the Assist in line with local hospital protocol. Do not bleach.

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

Symbol glossary available at www.gbukbanana.com.

GBUK Group Ltd.
Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0) 1757 288 587
US: 18665 289 422
info@gbukbanana.com

QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

B-Assist-03 2301 04