FloJac™

USER GUIDE

		Page
EN	FloJac™	4-9
FR	FloJac™	10-11
IT	FloJac™	12-13
ES	FloJac™	14-15
DE	FloJac™	16-17
DA	FloJac™	18-19
NO	FloJac™	20-21
SV	FloJac™	22-23
NL	FloJac™	24-25
PL	FloJac™	26-27
PT	FloJac™	28-29
EL	FloJac™	30-31
TR	FloJac™	32-33
CS	FloJac™	34-35
ET	FloJac™	36-37
HU	FloJac™	38-39
LV	FloJac™	40-41
MT	FloJac™	42-43
RO	FloJac™	44-45
SL	FloJac™	46-47

For more information on the B-FloJac39, please see accompanying IFU.

This user guide covers all the devices intended to be used with the FloJac, availability is dependent on territory.

Contents:

B-FloTrolley-CS
B-FloPump Air Supply
FM-CS139 Reusable Air Mattress
Discovery 1000 Lithium Battery Powerpack
EvacPRO+ Evacuation Support Device
B-FlexRB Flexible Roll Board

B-FloTrolley-CS

Intended Use: The B-FloTrolley-CS is intended to be used for the storage and transportation of the following items:

Code	Description	Supplied in Kits
B-FloJac39	FloJac™ Air Assisted Flat Lifting Device - 39" Wide	Bronze/Silver/Gold
B-FloPump	Air Transfer Pump	Bronze/Silver/Gold
FM-CS139	39" Flo Reusable Mattress	Silver/Gold
Discovery 1000	Power Pack	Gold
B-EVACPRO	3ET Evacuation Pro+	Gold
B-FlexRB	Flexible Roll Board	Bronze

Product Support:

Should you require support on the B-FloTrolley-CS please call +44 (0)1757 288 587, quoting the LOT number of the device, this can be found on the UDI Label on the IFU Pocket. See Fig 2.

Additional items - Ferno Lifting Scoop Stretcher

The Ferno Lifting Scoop Stretcher is not supplied as part of the FloJac Kit but the B-FloTrolley-CS can transport this item See section Storing and transporting the Ferno Lifting Scoop Stretcher below).

Image and part identification Please refer to Fig 1, 2 & 3 (Page 6)

Fig 1

1. Front Storage Compartment
2. Front B-EVACPRO buckle strap
3. Front buckle strap pocket
4. Powerpack Compartment Front Access Door
5. Flip Over door (with Magnetic Closure).
6. Door Handle

Fig 2

7. Power Pack Frame
8. UDI Label and IFU Pocket
9. Rear plate
10. Main Storage compartment

Fig 3

11. Powerpack rear access mesh door.
12. FloJac Securement straps
13. Rear buckle strap pocket
14. Rear B-EVACPRO buckle strap
15. Rear Buckle Strap

FloTrolley-CS – Warnings and precautions

- Heavy! Do not lift. Remove all trolley contents before moving off the pallet.
- Always apply rear brakes to the wheels when stationary.
- The trolley should always be parked facing down when on an inclined surface.
- During transportation, ensure that all items are secured as per this user guide and the trolley door is closed.
- Only the B-EVACPRO should be stored on top of the trolley. It should be secured as per this user guide.
- The trolley should ONLY be manoeuvred by the handle using both hands.
- The trolley is intended to be used with the items referenced in this user guide.

- DO NOT stand in or on the trolley.
- If using a Discovery 1000 Power Pack also ensure it is seated correctly in the Power Pack Frame.
- If using the powerpack in ambient temperatures above 20 degs C ensure the Powerpack Front Access Door is open to aid cooling.
- When transporting a Ferno Lifting Scoop Stretcher ensure this is secure as per section Storing and transporting the Ferno Lifting Scoop Stretcher below.

Operating Instructions

Refer to Fig 1-10 on page 6-7

Front Storage Compartment - See Fig 5: is intended to store the following items:

- Slide and Transfer Sheets
- FM-CS139 39" Flo Reusable Mattress
- B-FloPump

Powerpack Compartment – See Fig 2

- Discovery 1000 Power Pack & Mains Charger
- If not using a Discovery 1000 Power Pack, this compartment can be used for miscellaneous items.

Main Storage compartment – See Fig 4

- B-Flojac39
- B_FLEXRB

The B-Flojac39 should be secured using the red clip straps shown in Fig 9. These clip straps can be locked to improve security see fig 10.

Storing and transporting the B-EVACPRO – See Fig 6a & 6b.

Position 1 – Ensure the B-EVACPRO is attached with both buckle straps as shown in Fig 6a.

Position 2 – Ensure that the B-EVACPRO is seated on the rear plate of the trolley, and secured using both the rear buckle straps as shown in Fig 6b.

Trolley Security

- Padlocks can be attached to the zipped doors and compartments to improve security of the trolley.

Storing and transporting the Ferno Lifting Scoop Stretcher - See Fig 7 & 8.

- When transporting a Ferno Lifting Scoop Stretcher always ensure that its secured using the Rear Buckle Strap.
- Ensure that the rear buckle strap is passed through the side handholds of the Ferno Lifting Scoop Stretcher.
- The Ferno Lifting Scoop Stretcher should be folded before storing and transporting on the trolley.
- Ensure that the Ferno Lifting Scoop Stretcher is orientated with red coupling at the top as shown in Fig 7.
- The rear buckle straps feature hoop rings that can be locked together to improve the security of Ferno Lifting Scoop Stretcher – See Fig 8

Accessing the Discovery 1000 Power Pack – See Fig 2.

1. When using the Discovery 1000 Power Pack in the B-FloTrolley-CS it should be accessed through the Powerpack compartment front access door.
2. To remove the Discovery 1000 Power Pack, ONLY use the powerpack rear access mesh door.
3. When placing the Discovery 1000 Power Pack into the compartment through the rear access mesh door and ensure it is seated securely in the powerpack frame.

Storage and handling

Keep dry and out of direct sunlight.

Cleaning

- In between uses, thoroughly clean the device as soon as possible using alcohol wipes or Actichlor. Ensure the cleaning solutions are non-abrasive.
- manually wipe down the device until the device is visibly clean.
- If the device does not appear visibly clean, repeat the cleaning step.
- Do not use bleach-based cleaners on the trolley, or any other part of the kit.

Maintenance

The Trolley should be periodically inspected to ensure the following:

- Brakes (rear wheels) work correctly
- NO damage to the frame

Disposal procedures

Dispose of device in line with local hospital policy.

B-FloTrolley-CS Product Dimensions:

LWH – 1175mm x 555mm x 960mm

Warranty

1. Warranty term

The B-FloTrolley-CS is warranted to be free from defects in materials and workmanship for (1) one year from date of shipment of the Device to you (the Warranty Period).

2. What to do if a defect arises

In the unlikely event that a problem arises within the Warranty Period, you must contact GBUK Banana on +44 (0)1757 288587 as soon as possible. You must not continue to use the Device once you have identified that there is an issue with it. On request we will promptly provide a temporary Device to ensure that your down time without a device is as short as possible. You must return the Device to us as soon as possible. The address for returns is: GBUK Banana, Woodland House, Blackwood Hall Business Park, North Duffield, Selby, North Yorkshire, YO8 5DD, UK. You must pay the cost of returning the Device to us.

3. What GBUK will do

On receipt of your Device, we will, at our option, either:

- repair your Device; or
- replace the Device if we feel that it cannot be repaired.

We will return the repaired or replacement Device to you at our cost. On receipt you must promptly (and in any event within 7 days) return the loan Device to the address set out in paragraph 2 at your cost.

4. Exclusions to the warranty

Please note that the following actions will invalidate the warranty:

- (a) if you make further use of the Device after notifying GBUK that there is a problem with the Device;
- (b) if the defect arises because you failed to follow GBUK's oral or written instructions as to the storage, commissioning, use or maintenance of the Device;
- (c) you alter or repair or permit any third party to alter or repair the Device without the consent of GBUK; and/or
- (d) The defect arises as a result of fair wear and tear, wilful damage, negligence, mishandling or abnormal storage or working conditions.

5. Warranty is personal

This warranty is personal to the person or organisation who purchased the Device and cannot be transferred to any other person without the prior written consent of GBUK.

A person who is not a party to this warranty shall not have any rights to enforce its terms.

6. Liability

6.1 GBUK will have no liability to you or any third party for any loss or damage caused if you continue to use the Device after identifying a fault with it.

6.2 Nothing in this warranty shall limit or exclude GBUK's liability for: (a) death or personal injury caused by its negligence or the negligence of its employees, agents or subcontractors (as applicable);

(b) fraud or fraudulent misrepresentation;

(c) breach of the terms implied by section 12 of the Sale of Goods Act 1979;

(d) defective products under the Consumer Protection Act 1987; or (e) any matter in respect of which it would be unlawful for GBUK to exclude or restrict liability.

6.3 Subject to paragraph 6.2:

(a) GBUK shall under no circumstances whatever be liable to the Customer, whether in contract, tort (including negligence), breach of statutory duty or otherwise, for any loss of profit, or any indirect or consequential loss arising under or in connection with this warranty; and

(b) GBUK's total liability to the Customer in respect of all other losses arising under or in connection with this warranty whether in contract, tort (including negligence), breach of statutory duty or otherwise shall in no circumstances exceed L1,000,000.

6.4 GBUK's liability under this clause 6 ceases on expiry of the Warranty Period.

6.5 This warranty sets out GBUK's entire responsibility for defects in the Device. All other conditions, warranties or other terms which might have effect between the parties or be implied or incorporated into this licence or any collateral contract, whether by statute, common law or otherwise, are hereby excluded, including the implied conditions, warranties or other terms as to satisfactory quality, fitness for purpose or the use of reasonable skill and care.

7. Governing Law and Jurisdiction

a) The Contract, and any dispute or claim arising out of or in connection with it or its subject matter or formation (including non-contractual disputes or claims), shall be governed by and construed in accordance with the law of England and Wales.

(b) Each party irrevocably agrees that the courts of England and Wales shall have exclusive jurisdiction to settle any dispute or claim arising out of or in connection with this Contract or its subject matter or formation (including noncontractual disputes or claims).

Fig 1

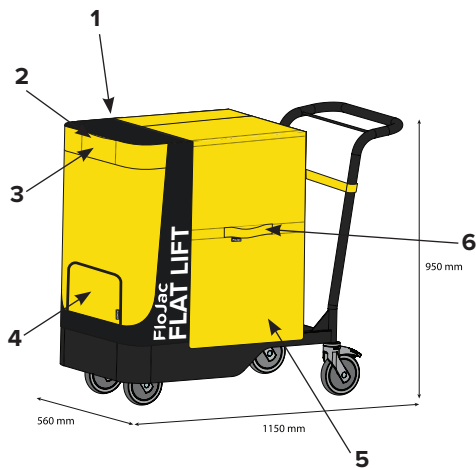
Fig 2

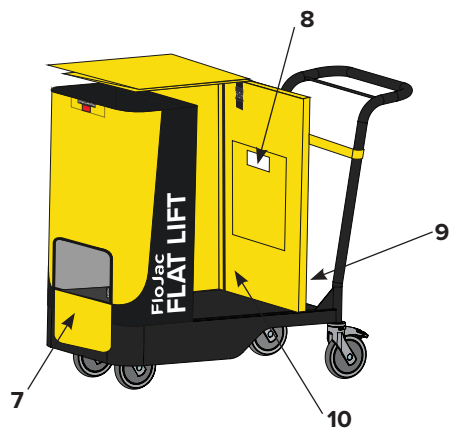
Fig 3

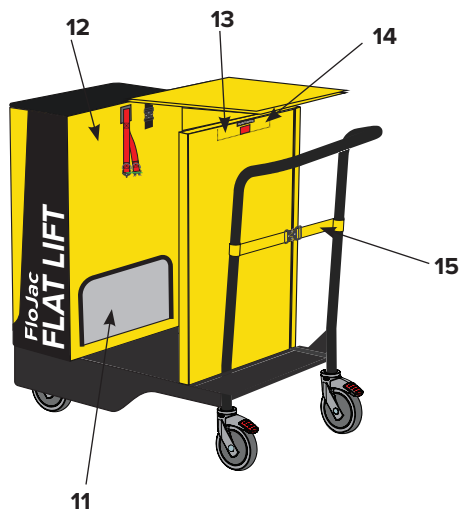
Fig 4

Fig 5

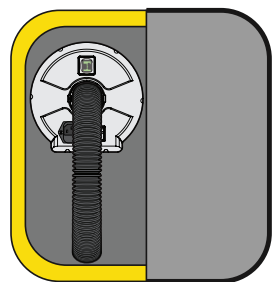
Fig 6a

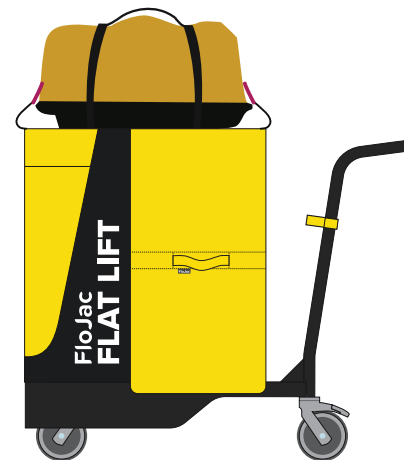
Fig 6b

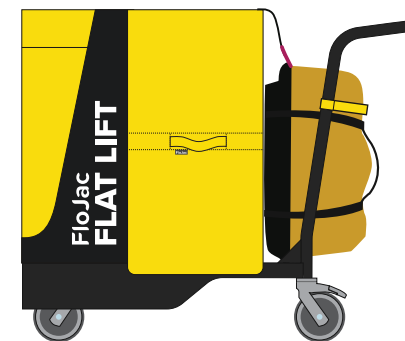
Fig 7

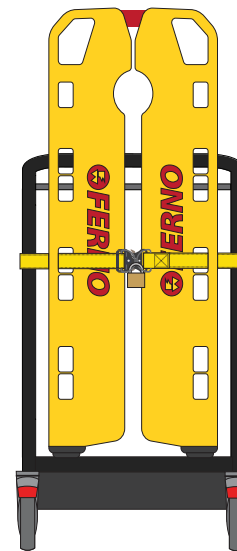
Fig 8

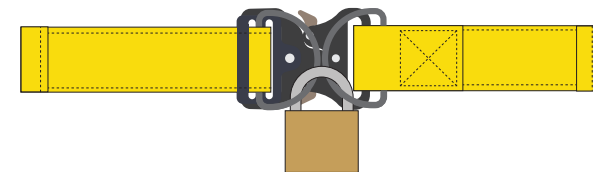
Fig 9



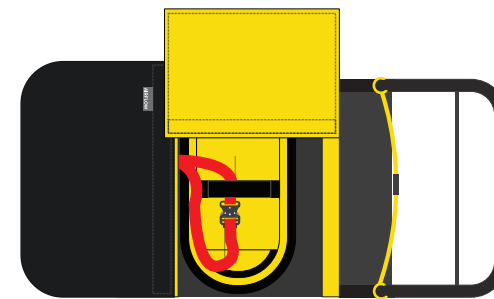
Fig 10

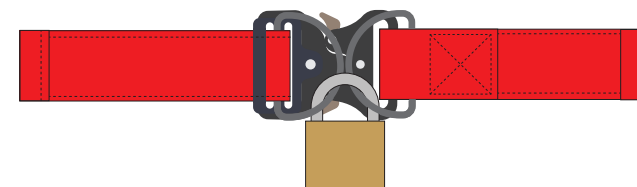B-FloPump Air Supply

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

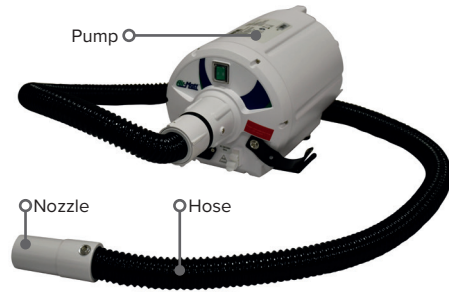The B-FloPump is intended to be used to inflate the FloJac and the Reusable Air Mattress (availability dependant on territory).

To use the B-FloPump:

1. Plug the power cord into the B-FloPump and then into the electrical outlet on the wall.
2. Attach the flexible hose into the B-FloPump.

With the FloJac:

3. Turn on B-FloPump so air is flowing, then press the nozzle onto the inlet valve.
4. Hold hose nozzle to the valves on each layer of the FloJac, inflate each layer individually starting at the base layer. For further instruction see FloJac IFU.
5. No automatic pressure relief exists on the B-FloPump. If pressure relief is desired, turn off B-FloPump or disconnect air hose nozzle and open inlet valve on FloJac.

With the Reusable Air Mattress:

6. Attach the hose nozzle into the mattress inlet port. Select the port that will allow travel of the hose without binding. Inlet ports are located at foot end of the mattress.
7. Use the snap to secure air hose to mattress.
8. No automatic pressure relief exists on device. If pressure relief is desired, turn off B-FloPump or disconnect air hose nozzle.

FM-CS139 Reusable Air Mattress

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

The Reusable Air Mattress is designed to be used with B-FloPump to allow clinicians to laterally transfer between surfaces and the FloJac.

The Reusable Air Mattress has a maximum patient weight limit of 450kg (1,000lbs).

To use the Reusable Air Mattress with the FloJac:

1. These instructions are general instruction for a lateral transfer, please follow manufacturer's instructions and local manual handling guidelines for all steps.
2. Position the device beneath the patient and centre the patient on the device, with the patient's head at the 'head' end.
3. Loosely fasten the patient safety straps across the patient to allow for the mattress to inflate.
4. Connect the B-FloPump with the mattress and follow the B-FloPump instruction to inflate the device.
5. Once the mattress is fully inflated, follow your local hospital manual handling policy to complete a lateral transfer from one surface onto the FloJac.
6. Once transfer is complete turn off the B-FloPump, unplug power cord from wall and device, disconnect hose and undo safety straps.
7. Remove device from under patient.

Discovery 1000 Lithium Battery Powerpack

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

The Discovery 1000 is intended to provide a portable source of power for the B-FloPump to inflate the FloJac and Reusable Air Mattress (availability dependant on territory).

To use the Discovery 1000:

1. Ensure the device has sufficient charge.
2. Connect the B-FloPump to the Discovery 1000.

With the FloJac:

3. Inflate the FloJac with the B-FloPump as referenced in the instructions above.

With the Reusable Air Mattress:

4. Inflate the Air Mattress with the B-FloPump as referenced in the instructions above.

WARRANTY SERVICE

Unless purchased with an extended warranty, your Powerpack 1200R is supplied with a 12-month warranty from the date of purchase against defects in material and workmanship under normal use and service. Should your Powerpack 1200R prove defective within 1 year from the date of purchase, please contact us for a returns number quoting the serial number and return the product, along with a copy of the proof of purchase and an explanation of the fault.

We do not accept liability for any 3rd party damage however so caused. Under this warranty we will replace or repair any parts found to be defective due to manufacturers defect.

We are not responsible in any way for damages, losses, or inconvenience caused by equipment failure or by user negligence, abuse, or use not accordance with the "User Manual" or any other additional safety, use, or warnings included in the product materials. Further, we are not liable for incidental, data loss, or consequential damages of any nature resulting from the use of this product and any liability shall not exceed the purchase price of the products.

EvacPRO+ Evacuation Support Device

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

The EvacPRO+ is an ultra-strong evacuation pad that works alongside the FloJac, safety tested to over 880lbs.

To use the EvacPRO+ with the FloJac:

1. When required, take the EvacPRO+ to the patient, undo the two securing webbing straps and holding the carry handle allow the EvacPRO+ to fall and un-roll out of the foot pocket. Undo all the securing webbing straps and lay flat.
2. The patient can then be transferred onto the EvacPRO+ by using the handles of the FloJac and sliding into position.
3. Ensure the head section is placed on the marked "HEAD" pad and if required insert some air into layer 5 of the FloJac to maintain the head and shoulders in a comfortable inclined position.
4. Secure the patient in place on the EvacPRO+ using the 4 x colour coded cross straps and then the two diagonal straps.
5. The length of the EvacPRO+ can then be adjusted to suit the patient by rolling up the foot pocket towards the head end and securing it with the 2 webbing straps fitted with easy adjust cam buckles.
6. The patient can then be moved using the pulling ladder handle strap at the head end of the EvacPRO+ and the side ladder handles as required. Additional red webbing lifting handles are positioned along the length of the EvacPRO+ and at either end there are yellow heavy duty anchor loops to be used as handles (additional ladder handles from foot pocket can be attached to these if required) or for attaching safety lines to assist in the safe movement of the EvacPRO+ up or down a slope. If moving on stairs, always pull the EvacPRO+ feet first down the stairs.

WARRANTY SERVICE

Standard Product Warranty

3ET Limited will repair or replace any 3ET manufactured product covered by this warranty for a period of 2 years from date of purchase against any material or manufacturing defect.

General Conditions of Warranty

Any warranty claim will be subject to the products being visually inspected / examined and if deemed necessary tested by 3ET or their designated representative. Products deemed to have excessive wear & tear or which have been used for other than their designed / designated purpose or have been maintained outside their prescribed testing and cleaning procedures, will be excluded from this warranty.

3ET Warranty Disputes

If in the event of a 3ET warranty claim dispute, 3ET limited agrees to be bound by an independent third-party decision on any disputed warranty claim.

B-FlexRB Flexible Roll Board

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

The Flexible Roll Board is intended to be used to transfer a patient from one surface onto the FloJac.

To use the Flexible Roll Board with the FloJac:

1. These instructions are general for a lateral transfer, please follow manufacturer's instructions and local manual handling guidelines for all steps.
2. The patient should roll onto their side with assistance from care givers.
3. The Flexible Roll Board should be placed about an inch away from the spine, either under the bed sheet/slide sheet or directly under the patient.
4. Ensure the Flexible Roll Board is placed under the head and heels of the patient.
5. Once the Flexible Roll Board is in position, the patient can be laterally transferred onto the FloJac.
6. Once the patient has been transferred, the Flexible Roll Board can be removed and stored in the bag.

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FloJac is a trade mark of **GBUK Group Ltd.**, registered in the UK and other countries and regions.
GBUK Group Ltd.
GBUK GmbH
Swiss AR Services AG