GBUKBanana
Part of the GBUK Group ●●●●

FloJac™

INSTRUCTIONS FOR USE

	Page
EN FloJac	4
FR FloJac	6
IT FloJac	9
ES FloJac	12
DE FloJac	14
DA FloJac	17
NO FloJac	20
SV FloJac	22
NL FloJac	25
PL FloJac	28
PT FloJac	31
EL FloJac	34
TR FloJac	37
CS FloJac	39
ET FloJac	42
HU FloJac	44
LV FloJac	47
MT FloJac	50
RO FloJac	52
SL FloJac	55

GBUK Group Ltd.
Woodland House, Blackwood Hall Business Park,
North Duffield, Selby, North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587 | US: 18665 289 422
www.gbukgroup.com | info@gbukgroup.com

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EU REP QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

CH REP Swiss AR Services AG
Industriestrasse 47
6300 Zug, Switzerland

FloJac

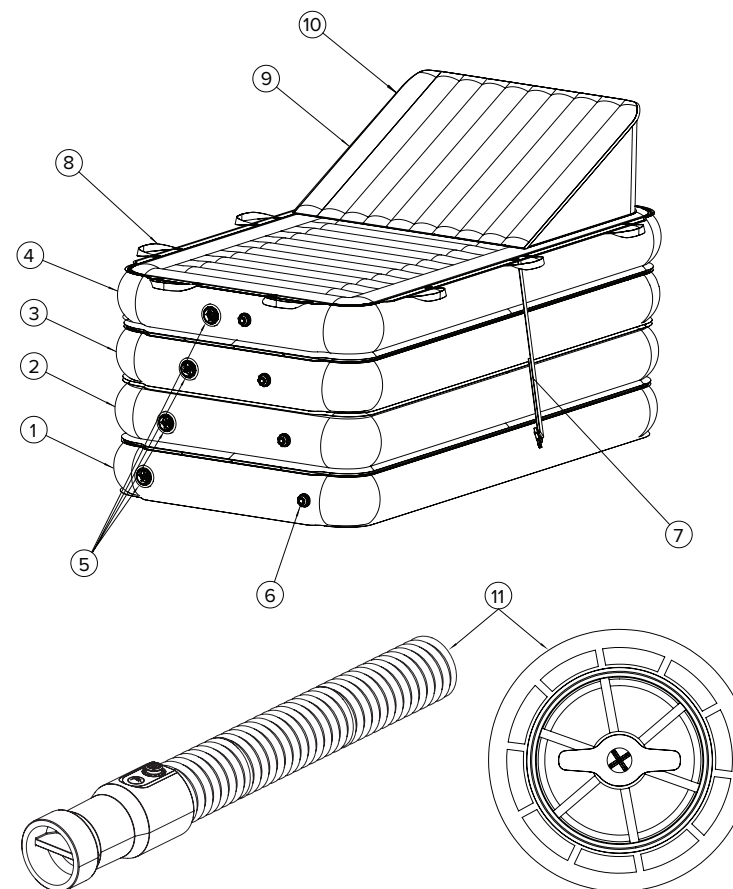
Image and Part Identification

1. Layer 1
2. Layer 2
3. Layer 3
4. Layer 4
5. Inlet valves 1-4
6. Pressure Indicator valve on each layer
7. Patient Safety Strap (attached on both sides)
8. Handles
9. Wedge support
10. Inlets and outlet valves for wedge support
11. Nozzle and inlet valve

EN

Should you require support on these products please call +44 (0)1757 288 587, quoting the LOT number of the device.

Image and Part Identification

1. Layer 1
2. Layer 2
3. Layer 3
4. Layer 4
5. Inlet valves 1-4
6. Pressure Indicator valve on each layer
7. Patient Safety Strap (attached on both sides)
8. Handles
9. Wedge support
10. Inlets and outlet valves for wedge support
11. Nozzle and inlet valve

Intended use

The device is intended to help clinicians safely lift patients from the floor.

Indications for use

The device is indicated for use on patients who require assistance in being lifted from the floor to another surface.

Description

The Banana FloJac is an air-assisted lifting device comprising of four inflatable layers and a wedge support. The wedge support can be used to provide comfort for the patient who may have respiratory difficulties and allows them to lie in a semi-recumbent position.

The FloJac has a safe working load (SWL) of 500 kg.

Patient target group

Patients requiring assistance in completing a vertical lift.

Intended users

The device is intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

Contra-indications

Patients with thoracic, cervical, or lumbar fractures that are deemed unstable unless a clinical decision and risk assessment has been performed.

Patients who exceed the SWL.

Do not use in direct contact with broken skin.

Warnings and precautions

- Do not exceed the SWL.
- Never leave a patient unattended on the FloJac.
- Carry out a full risk assessment prior to using the FloJac with every patient to assess what assistance may be required e.g., spinal board.
- Always inflate the layers in order (1 to 4, and finally the wedge support). DO NOT inflate the wedge support first. Layers must be fully inflated in sequence. If not, this could cause the FloJac to become unstable.
- Stop inflating layer when pressure indicator releases air. DO NOT OVER INFLATE. To detect air release from pressure indicator, place hand directly over the pressure indicator of the corresponding layer.
- To avoid explosion, do not use in the presence of anaesthetic gas. Do not use in hyperbaric chambers or oxygen tents.
- Assess the surrounding area prior to manoeuvring the FloJac. Avoid contact with sharp objects or implements.

- The FloJac can be moved/rotated using the ten handles situated around the device but should never be used to transport a patient down a corridor or across a car park. DO NOT use the patient safety straps to move the FloJac.
- Ensure safety straps are tightened before moving or turning. At least two trained and experienced caregivers are required to move the device. Each caregiver MUST use two handles at one time whilst manoeuvring the patient.
- Do not attempt to move or rotate a patient on an uninflated FloJac.
- Refer to B-FloPump instructions for use when using the pump.
- Inspect the device. The material must be intact and not exhibit any splits or tears. The stitching on the handles should be inspected to ensure the stitching is intact.
- Do not use the device if it appears worn damaged or, wet
- Ensure that the correct device is being used
- Always use appropriate equipment and sizes
- Follow RCN (or equivalent) guidelines
- In the event of a fire ensure the FloJac is concealed in the EVAC Pro for evacuation

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

Compatibility

Please visit <https://www.gbukgroup.com/flojac.pdf> for a list of compatible pumps.

User Guide

Please visit <https://www.gbukgroup.com/flojac.pdf> for the FloJac User Guide.

Operating instructions

1. Unclip the two front buckles on the storage bag and unfold the flap.
2. Unclip the single buckle to release the yellow section of the bag.
3. Unroll the FloJac onto the floor.
4. Ensure all inlet valves are closed before inflating the device. Ensure that all five inlet valves are closed on the device. If these are not closed, twist the round valve handle in an anti-clockwise direction.
5. Plug the pump into a nearby socket or powerpack. Ensure that the wires are not a trip hazard.
6. Transfer the patient onto the FloJac using slide sheets, an air mattress, or a flexible roll board. Ensure that the patient's head is at the head end of the device as indicated by the head icon, and that the patient is in the centre of the FloJac.
7. Secure the patient on the FloJac by connecting the safety straps. DO NOT tighten the straps across the patient's body.

INFLATING THE FLOJAC

1. Starting with Layer 1, press on/off button on the compressor and inflate this layer; this will inflate within seconds. The inlet valves will open once the hose nozzle (with adapter) is pushed onto the inlet valve.
2. Once the hose nozzle is removed, the inlet valve will close.
3. If the layer is over-inflated, the pressure indicator valves will start to expel air. This is a safety feature of this device, but the hose must be removed from the layer immediately.
4. Repeat steps 1 – 3 for layers 2, 3 and 4 or until the FloJac reaches the desired height.
5. The inlet valve for the wedge support is at the head end of the device and this can be inflated if required to form a wedge support. Follow steps 1 – 3 for inflation.

6. Once all the required layers have been inflated, turn off the compressor and tighten the safety straps
7. The patient trolley or bed can now be brought alongside the FloJac to enable the patient to be laterally transferred the steps in the operating instructions above.
8. There are 3 handles on each side, 2 at the head/foot end which can be used to change orientation of the device. DO NOT transport a patient down a corridor or across a car park.

DEFLATING THE FLOJAC

1. Once the patient has been transferred from the FloJac, it can now be deflated. Deflate the wedge support first (if inflated) then layers 4, 3, 2 and 1.
2. To deflate the device push inlet valve in and turn to the right.
3. Once deflated, fold the FloJac in half lengthways and roll from head end to foot end.
4. The FloJac can now be placed into the storage bag and be placed back into the trolley.
5. Please ensure that all layers and wedge are fully deflated before folding and storing into the trolley.

Storage and handling

Keep dry and out of direct sunlight.

Disposal procedures

Dispose of device in line with local hospital policy.

Materials:

Top layer & wedge support: Nylon. Tested to EN597-1

Handles: TPU

Not made with natural rubber latex.

Length: 77"/195.5cm

Width: 39"/99cm

Height of layer 1 to 4 when fully inflated: 30"

Warranty

1. Warranty term

The FloJac patient lift kit (the Device) is warranted to be free from defects in materials and workmanship for (1) one year from date of shipment of the Device to you (the Warranty Period).

2. What to do if a defect arises

In the unlikely event that a problem arises within the Warranty Period, you must contact GBUK Banana on +44 (0)1757 288587 as soon as possible. You must not continue to use the Device once you have identified that there is an issue with it. On request we will promptly provide a temporary Device to ensure that your down time without a device is as short as possible. You must return the Device to us as soon as possible. The address for returns is: GBUK Banana, Woodland House, Blackwood Hall Business Park, North Duffield, Selby, North Yorkshire, YO8 5DD, UK. You must pay the cost of returning the Device to us.

3. What GBUK will do

On receipt of your Device, we will, at our option, either:

- repair your Device; or
- replace the Device if we feel that it cannot be repaired.

We will return the repaired or replacement Device to you at our cost. On receipt you must promptly (and in any event within 7 days) return the loan Device to the address set out in paragraph 2 at your cost.

4. Exclusions to the warranty

Please note that the following actions will invalidate the warranty:

- (a) if you make further use of the Device after notifying GBUK that there is a problem with the Device;

- (b) if the defect arises because you failed to follow GBUK's oral or written instructions as to the storage, commissioning, use or maintenance of the Device;
- (c) you alter or repair or permit any third party to alter or repair the Device without the consent of GBUK; and/or (d) the defect arises as a result of fair wear and tear, wilful damage, negligence, mishandling or abnormal storage or working conditions.

5. Warranty is personal

This warranty is personal to the person or organisation who purchased the Device and cannot be transferred to any other person without the prior written consent of GBUK.

A person who is not a party to this warranty shall not have any rights to enforce its terms.

6. Liability

6.1 GBUK will have no liability to you or any third party for any loss or damage caused if you continue to use the Device after identifying a fault with it.

6.2 Nothing in this warranty shall limit or exclude GBUK's liability for:

- (a) death or personal injury caused by its negligence or the negligence of its employees, agents or subcontractors (as applicable);
- (b) fraud or fraudulent misrepresentation;
- (c) breach of the terms implied by section 12 of the Sale of Goods Act 1979;
- (d) defective products under the Consumer Protection Act 1987; or
- (e) any matter in respect of which it would be unlawful for GBUK to exclude or restrict liability.

6.3 Subject to paragraph 6.2:

- (a) GBUK shall under no circumstances whatever be liable to the Customer, whether in contract, tort (including negligence), breach of statutory duty or otherwise, for any loss of profit, or any indirect or consequential loss arising under or in connection with this warranty; and
- (b) GBUK's total liability to the Customer in respect of all other losses arising under or in connection with this warranty whether in contract, tort (including negligence), breach of statutory duty or otherwise shall in no circumstances exceed £1,000,000.

6.4 GBUK's liability under this clause 6 ceases on expiry of the Warranty Period.

6.5 This warranty sets out GBUK's entire responsibility for defects in the Device. All other conditions, warranties or other terms which might have effect between the parties or be implied or incorporated into this licence or any collateral contract, whether by statute, common law or otherwise, are hereby excluded, including the implied conditions, warranties or other terms as to satisfactory quality, fitness for purpose or the use of reasonable skill and care.

7. Governing Law and Jurisdiction

- (a) The Contract, and any dispute or claim arising out of or in connection with it or its subject matter or formation (including non-contractual disputes or claims), shall be governed by and construed in accordance with the law of England and Wales.
- (b) Each party irrevocably agrees that the courts of England and Wales shall have exclusive jurisdiction to settle any dispute or claim arising out of or in connection with this Contract or its subject matter or formation (including non-contractual disputes or claims).

Cleaning

- In between uses, thoroughly clean the device as soon as possible using alcohol wipes or Actichlor. Ensure the cleaning solutions are non-abrasive.
- Inflate all layers fully and manually wipe down the device until the device is visibly clean.
- If the device does not appear visibly clean, repeat the cleaning step. Allow the device to air dry fully before storing.
- If the device remains contaminated upon visual inspection following multiple cleaning attempts, dispose of the device in line with local hospital guidelines.
- Do not use bleach-based cleaners on the FloJac, trolley, or any other part of the kit.

Maintenance

The FloJac should be periodically inspected to ensure the following:

- All valves are in full working order and are able to be locked open and closed.
- All patient safety straps and buckles are in full working order.
- Side handles are all attached and fully secured.
- There are no punctures or tears in the FloJac.

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

Symbol glossary available at www.gbukgroup.com
Not made with natural rubber latex.

Caution: Federal law restricts this device to sale by or on the order of a manual handler, trained healthcare professional and others appropriately trained in use of the device.

FloJac is a trade mark of GBUK Group Ltd., registered in the UK and other countries and regions.

GBUK Group Ltd.

Woodland House,
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK
EU: +44 (0)1757 288 587

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QualRep Services BV
Utrechtseweg 310 B42
6812 AR Arnhem,
The Netherlands

Swiss AR Services AG
Industriestrasse 47
6300 Zug, Switzerland

B-FloJac-03 2506 10

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GBUK GmbH