Into Prone™ and Orbit™

INSTRUCTIONS FOR USE

	Page
EN Into Prone™ and Orbit™	4
FR Into Prone™ and Orbit™	5
IT Into Prone™ and Orbit™	6
ES Into Prone™ and Orbit™	7
DE Into Prone™ and Orbit™	8
DA Into Prone™ and Orbit™	9
NO Into Prone™ and Orbit™	10
SV Into Prone™ and Orbit™	11
NL Into Prone™ and Orbit™	12
PL Into Prone™ and Orbit™	13
PT Into Prone™ and Orbit™	14
EL Into Prone™ and Orbit™	15
TR Into Prone™ and Orbit™	16
CS Into Prone™ and Orbit™	17
ET Into Prone™ and Orbit™	18
HU Into Prone™ and Orbit™	19
LV Into Prone™ and Orbit™	20
MT Into Prone™ and Orbit™	21
RO Into Prone™ and Orbit™	22
SL Into Prone™ and Orbit™	23

Into Prone™ and Orbit™

Instructions For Use

Please read these instructions before use.

Intended use

The Into Prone and Orbit is intended to help caregivers turn patients into prone position. The Orbit is additionally intended to help caregivers turn patients out of prone position.

Indications for use

The Into Prone and Orbit are indicated for use on patients who require assistance to turn into prone position. The Orbit is additionally indicated for use on patients who require assistance to turn out of prone position.

Description

The device allows caregivers to maintain control during the entire turn. These devices have a Safe Working Load (SWL) limit of 130kg.

Product Code	Product Description
B-Orbit	Banana Orbit- Into and Out of Prone 90cm x 195cm
IP090195PRN	Banana Into Prone 90cm x 195cm

Patient target group

Patients who require assistance in proning.

Intended users

These devices are intended to be used by trained healthcare professionals or those with the appropriate training or knowledge.

Contra-indications

Do not use in direct contact with broken skin.
Do not use with patients who exceed the SWL limit.

Warnings and precautions

Follow local manual handling policy from your hospital or Royal College of Nursing guidelines.

- Do not use the devices to lift a patient.
- Do not launder or bleach the device.
- For single patient use only. Cross patient use of this device poses a risk of infection and/or cross contamination.
- Do not exceed the 130kg SWL limit.

A risk assessment and visual inspection must be performed by competent person(s) before using the Orbit/Into Prone. This should consider:

- The physical and medical status of the patient, including skin integrity.
- The status of the device. The material must be intact and not exhibit any splits or tears. The stitching on the handles should be inspected to ensure the stitching is intact. Do not use the device if it appears damaged, wet, or worn.
- The suitability of the device.

In the event of a serious incident involving the device, the user should report the event to GBUK and the competent authority in which the user/patient is established. In the event of a suspected device fault, the user should report the event to GBUK, and the device should be returned to GBUK for examination.

Operating instructions

Follow your local manual handling policy for the below manoeuvre.

- Place the Into Prone (blue handles) onto the patient trolley.
- If using the Orbit (red handles), place on top of the gel pads or Montreal type frame on the operating/receiving table.
- Position the patient on trolley with the Into Prone underneath.
- If required, place a GBUK Banana tubular slide sheet over the gel pad/frame. This is to allow for small adjustments to the patient's position.
- Position the trolley alongside the operating table and ensure both surfaces are locked into position.
- Raise the trolley height so the patient is positioned just above the pads/Montreal frame.
- Use the handles to position the patient, ensuring their shoulders are correctly positioned to act as a pivot during the proning movement.
- Pass the handle straps across the patient, interweaving the straps through the large loops on the opposite side of the device, pull the opposing straps to cocoon and secure the patient within the device.
- Ensure there are two members of staff at each side of the patient and one member of staff at the head.
- Roll the cocooned patient across from the trolley into prone position on the receiving bed/trolley. Ensure the movement is fully controlled by staff.
- Once the patient is in prone position, remove the device and slide sheet.
- If using the Orbit (red handles) to turn the patient out of prone, repeat Step 3 to Step 11 above.

Storage and handling

Keep dry and out of direct sunlight.

Disposal procedures

Follow your hospital protocol.

Cleaning

These devices are intended for single patient use. If lightly soiled, manually wipe down the sheet as soon as possible using alcohol-based cleaning solutions (up to 70%) or normal disinfectant solutions until the sheet is visibly clean. If the sheet does not appear visibly clean and remains contaminated upon visual inspection, discard the sheet in line with local hospital protocol.

Not made with natural rubber latex.

Symbol glossary available at www.gbukgroup.com

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Ni izdelano iz naravnega lateksa.

Slovor simbolov najdete na naslovu www.gbukgroup.com

This image shows a single sheet of white paper with horizontal ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

GBUK Group Ltd.

Woodland House
Blackwood Hall Business Park,
North Duffield, Selby,
North Yorkshire, YO8 5DD, UK

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Swiss AR Services AG

Industriestrasse 47,
6300 Zug, Switzerland

Caution: Federal law restricts this device to sale by or on the order of a manual handler, trained healthcare professional and others appropriately trained in use of the device.

Symbol glossary available at www.gbukgroup.com.
Not made with natural rubber latex.

Into Prone and Orbit are trade marks of GBUK Group Ltd., registered in the UK and other countries and regions.

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GBUK GmbH