**GBUKBanana**  
Part of the GBUK Group ●●●●

# FloJac™

USER GUIDE

**GBUK Group Ltd.**  
Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK  
**EU: +44 (0)1757 288 587**  
**US: 18665 289 422**  
[www.gbukbanana.com](http://www.gbukbanana.com)  
[info@gbukbanana.com](mailto:info@gbukbanana.com)

**EC REP** **QualRep Services BV**  
Utrechtseweg 310 B42  
6812 AR Arnhem,  
The Netherlands

**CH REP** **Swiss AR Services GmbH**  
Industriestrasse 47  
6300 Zug, Switzerland

	<b>Page</b>
<b>EN</b> FloJac™	4
<b>FR</b> FloJac™	7
<b>IT</b> FloJac™	10
<b>ES</b> FloJac™	13
<b>DE</b> FloJac™	16
<b>DA</b> FloJac™	19
<b>NO</b> FloJac™	22
<b>SV</b> FloJac™	25
<b>NL</b> FloJac™	28
<b>PL</b> FloJac™	31
<b>PT</b> FloJac™	34
<b>EL</b> FloJac™	37
<b>TR</b> FloJac™	40
<b>CS</b> FloJac™	43
<b>ET</b> FloJac™	46
<b>HU</b> FloJac™	49
<b>LV</b> FloJac™	52
<b>MT</b> FloJac™	55
<b>RO</b> FloJac™	58
<b>SL</b> FloJac™	61

EN

For more information on the B-FloJac39 and B-Trolley-C please see accompanying IFU. This user guide covers all the devices intended to be used with the FloJac, availability is dependent on territory.


**Contents:**

- B-FloPump Air Supply
- FM-CS139 Reusable Air Mattress
- B-FloPower Lithium Battery Powerpack
- EvacPRO+ Evacuation Support Device
- B-FlexRB Flexible Roll Board

**B-FloPump Air Supply**

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

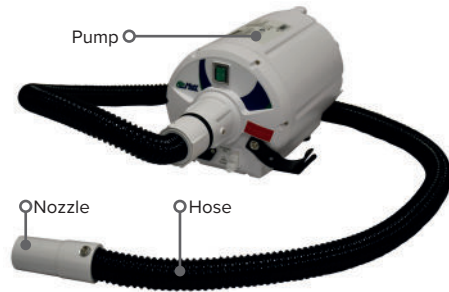The B-FloPump is intended to be used to inflate the FloJac and the Reusable Air Mattress (availability dependant on territory).

**To use the B-FloPump:**

1. Plug the power cord into the B-FloPump and then into the electrical outlet on the wall.
2. Attach the flexible hose into the B-FloPump.

**With the FloJac:**

3. Turn on B-FloPump so air is flowing, then press the nozzle onto the inlet valve.
4. Hold hose nozzle to the valves on each layer of the FloJac, inflate each layer individually starting at the base layer. For further instruction see FloJac IFU.
5. No automatic pressure relief exists on the B-FloPump. If pressure relief is desired, turn off B-FloPump or disconnect air hose nozzle and open inlet valve on FloJac.

**With the Reusable Air Mattress:**

6. Attach the hose nozzle into the mattress inlet port. Select the port that will allow travel of the hose without binding. Inlet ports are located at foot end of the mattress.
7. Use the snap to secure air hose to mattress.
8. No automatic pressure relief exists on device. If pressure relief is desired, turn off B-FloPump or disconnect air hose nozzle.

**FM-CS139 Reusable Air Mattress**

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

The Reusable Air Mattress is designed to be used with B-FloPump to allow clinicians to laterally transfer between surfaces and the FloJac.

The Reusable Air Mattress has a maximum patient weight limit of 450kg (1,000lbs).

**To use the Reusable Air Mattress with the FloJac:**

1. These instructions are general instruction for a lateral transfer, please follow manufacturer's instructions and local manual handling guidelines for all steps.
2. Position the device beneath the patient and centre the patient on the device, with the patient's head at the 'head' end.
3. Loosely fasten the patient safety straps across the patient to allow for the mattress to inflate.
4. Connect the B-FloPump with the mattress and follow the B-FloPump instruction to inflate the device.
5. Once the mattress is fully inflated, follow your local hospital manual handling policy to complete a lateral transfer from one surface onto the FloJac.
6. Once transfer is complete turn off the B-FloPump, unplug power cord from wall and device, disconnect hose and undo safety straps.
7. Remove device from under patient.

**B-FloPower Lithium Battery Powerpack**

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

The B-FloPower is intended to provide a portable source of power for the B-FloPump to inflate the FloJac and Reusable Air Mattress (availability dependant on territory).

**To use the B-FloPower:**

1. Ensure the device has sufficient charge.
2. Connect the B-FloPump to the B-FloPower.

**With the FloJac:**

3. Inflate the FloJac with the B-FloPump as referenced in the instructions above.

**With the Reusable Air Mattress:**

4. Inflate the Air Mattress with the B-FloPump as referenced in the instructions above.

**WARRANTY SERVICE**

Unless purchased with an extended warranty, your Powerpack 1200R is supplied with a 12-month warranty from the date of purchase against defects in material and workmanship under normal use and service. Should your Powerpack 1200R prove defective within 1 year from the date of purchase, please contact us for a returns number quoting the serial number and return the product, along with a copy of the proof of purchase and an explanation of the fault.

We do not accept liability for any 3rd party damage however so caused. Under this warranty we will replace or repair any parts found to be defective due to manufacturers defect.

We are not responsible in any way for damages, losses, or inconvenience caused by equipment failure or by user negligence, abuse, or use not accordance with the "User Manual" or any other additional safety, use, or warnings included in the product materials. Further, we are not liable for incidental, data loss, or consequential damages of any nature resulting from the use of this product and any liability shall not exceed the purchase price of the products.

**EvacPRO+ Evacuation Support Device**

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

The EvacPRO+ is an ultra-strong evacuation pad that works alongside the FloJac, safety tested to over 880lbs.

**To use the EvacPRO+ with the FloJac:**

1. When required, take the EvacPRO+ to the patient, undo the two securing webbing straps and holding the carry handle allow the EvacPRO+ to fall and un-roll out of the foot pocket. Undo all the securing webbing straps and lay flat.
2. The patient can then be transferred onto the EvacPRO+ by using the handles of the FloJac and sliding into position.
3. Ensure the head section is placed on the marked "HEAD" pad and if required insert some air into layer 5 of the FloJac to maintain the head and shoulders in a comfortable inclined position.
4. Secure the patient in place on the EvacPRO+ using the 4 x colour coded cross straps and then the two diagonal straps.
5. The length of the EvacPRO+ can then be adjusted to suit the patient by rolling up the foot pocket towards the head end and securing it with the 2 webbing straps fitted with easy adjust cam buckles.
6. The patient can then be moved using the pulling ladder handle strap at the head end of the EvacPRO+ and the side ladder handles as required. Additional red webbing lifting handles are positioned along the length of the EvacPRO+ and at either end there are yellow heavy duty anchor loops to be used as handles (additional ladder handles from foot pocket can be attached to these if required) or for attaching safety lines to assist in the safe movement of the EvacPRO+ up or down a slope. If moving on stairs, always pull the EvacPRO+ feet first down the stairs.

**WARRANTY SERVICE**

**Standard Product Warranty**

3ET Limited will repair or replace any 3ET manufactured product covered by this warranty for a period of 2 years from date of purchase against any material or manufacturing defect.

**General Conditions of Warranty**

Any warranty claim will be subject to the products being visually inspected / examined and if deemed necessary tested by 3ET or their designated representative. Products deemed to have excessive wear & tear or which have been used for other than their designed / designated purpose or have been maintained outside their prescribed testing and cleaning procedures, will be excluded from this warranty.

### 3ET Warranty Disputes

If in the event of a 3ET warranty claim dispute, 3ET limited agrees to be bound by an independent third-party decision on any disputed warranty claim.

### B-FlexRB Flexible Roll Board

UK only

Please read any relevant manufacturer information before attempting to use this device with the FloJac.

The Flexible Roll Board is intended to be used to transfer a patient from one surface onto the FloJac.

#### To use the Flexible Roll Board with the FloJac:

1. These instructions are general for a lateral transfer, please follow manufacturer's instructions and local manual handling guidelines for all steps.
2. The patient should roll onto their side with assistance from care givers.
3. The Flexible Roll Board should be placed about an inch away from the spine, either under the bed sheet/slide sheet or directly under the patient.
4. Ensure the Flexible Roll Board is placed under the head and heels of the patient.
5. Once the Flexible Roll Board is in position, the patient can be laterally transferred onto the FloJac.
6. Once the patient has been transferred, the Flexible Roll Board can be removed and stored in the bag.

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GBUK Group award-winning specialist companies are experts in the provision of enteral feeding, critical care and patient moving and handling products and technologies. GBUK Group companies are suppliers to every UK NHS Trust and to a rapidly expanding export customer base.

GBUK Banana is a trading name of **GBUK Group Ltd.**

Symbol glossary available at [www.gbukbanana.com](http://www.gbukbanana.com)  
Not made with natural rubber latex.

FloJac is a trade mark of GBUK Group Ltd., registered in the UK and other countries and regions.

**GBUK Group Ltd.**

Woodland House,  
Blackwood Hall Business Park,  
North Duffield, Selby,  
North Yorkshire, YO8 5DD, UK  
**EU: +44 (0)1757 288 587**  
**US: 18665 289 422**  
[www.gbukbanana.com](http://www.gbukbanana.com)  
[info@gbukbanana.com](mailto:info@gbukbanana.com)

**QualRep Services BV**  
Utrechtseweg 310 B42  
6812 AR Arnhem,  
The Netherlands

**Swiss AR Services GmbH**  
Industriestrasse 47  
6300 Zug, Switzerland